## SOOTHERSACK® 16610, 17007

### DESCRIPTION

Soothersacks® are luxury wheat filled bands that contain carefully selected oils to calm and to relieve stress, tension, muscle spasm and insomnia. This product has been created and produced in England without any exploitation and uses only naturally occurring substances.

### CONTENTS

1 x Soothersack® with removable cover

### HEATING INSTRUCTIONS

#### 1. Microwave Method

Remove the corduroy cover and place the inner wheatsack in the microwave. Heat for two and a half minutes on a high power.

#### 2. Oven Method

Remove the corduroy cover and place the inner wheatsack in a casserole dish with a lid, or wrapped in tin foil, and place it in the oven. Heat for ten to fifteen minutes on a Low-Medium heat, (160°C or Gas Mark 3).

Do not leave your Soothersack® unattended whilst heating. Should you require more warmth do not exceed three minutes in the microwave and twenty minutes in the oven. If these times are exceeded overheating may occur and the Soothersack® may be damaged. The Soothersack® is designed to stay warm for up to an hour.

### AS A COLD COMPRESS

Put the inner wheat sack into a plastic bag and then into the freezer. For best results leave overnight. When removing the bag, brush off any ice which may have formed on the wheatsack. Place the inner wheatsack back inside the corduroy cover.

### TO REPLENISH THE OILS

After 3-6 months the oils will be need to be replenished. Place eight to ten drops of oil onto a still warm Soothersack®. Fold the sack in half and leave in a plastic bag overnight. Top-Up Oil is available from ROMPA® (16607/17008).

### TO WASH THE COVER

Remove the corduroy cover. Never allow the inner wheatsack to become wet. Wash deep colours separately. Wash inside out. Iron on the reverse. See the label on the inside of the cover.

### SAFETY

This product has been thoroughly tested by an independent Testing Laboratory approved of by the National Accreditation of Certification Bodies.

Seek medical advice before using this product during pregnancy.

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