



1. Product Name

Super Interactive Switch Box

2. Product Code

17274

3. Colour

As shown

4. Brief Description

This Super Interactive Switch Box controls the Maxi, Aquatic, Ball and Midi Bubble Tubes, Bubble Wall and ROMPA® Waterless Rainbow Tube. Choose the colours and the flow of the bubbles by pressing appropriate buttons.

5. Contents

- 1 x Interactive Switch

6. Snoezelen® Stimulations

- Sight

- Sound

7. Best Use

Use indoors, connected to ROMPA Maxi and Midi bubble tubes.

8. Compatible ROMPA® Products

This Super Interactive Switch Box can be used with the Maxi, Aquatic, Ball and Midi Bubble Tubes, Bubble Wall and ROMPA® Waterless Rainbow Tubes from ROMPA®. See www.rompa.com to view the full selection of sizes and widths of different bubble tubes as well as for switches.

9. Starting Up

- Connect to Bubble Tube/Wall
- Adjust settings on the underside of the Switch Box

10. Detailed Description

The controls on the underside of the Switch Box allow control over the speed of colour change as well as when the bubbles are activated. For further interactivity, this switch box can be used with different ROMPA® switches, giving the user nine different display variations. *Note: When used with a switch, the switch overrides the circular switching box provided.* There is a feedback click when the switches are pressed.

11. Safety

- Always supervise the use of this product.
- Install indoors in a dry environment.

12. Technical Specification

Size:	37cm Diameter X 12cmH
Material:	Latex Free
Power:	This switch draws power from the Bubble Tube/Wall and so does not require batteries or connection to the mains.

13. Installation

1. Switch off the Bubble Tube completely. Connect the 5 Button Switch Box to the RJ45



socket in the base of the Bubble Tube (the socket to the right of the SPEED dial). The protective cover will have to be removed first and stored in a safe place so it can be replaced after use.



RJ45 socket

2. To select a coloured light press one of the four buttons (red, green, yellow, blue) on the top. The selected colour will only be activated for a few seconds (the exact time depends on how you have set the SPEED and DELAY buttons) before the cycle of colours resumes. The selected colour will not remain indefinitely till another colour is selected using the switch box. Experiment with the SPEED and DELAY buttons until you find a setting that best suits your requirements. For example, set SPEED to a minimum to keep the colour change as slow as possible.
3. To stop the bubbles press the central button on the top. Release the button to make the bubbles start again. On the underside of the Switch Box there is a dial labelled BUBBLES with two options – NORMALLY ON and NORMALLY OFF. Switch between these so that one way the bubbles start if the central button is pressed and the other way the bubbles will stop if the central button is pressed.
4. To adjust the speed of the light change/flash rate use the SPEED and DELAY buttons underneath labelled COLOURS.

YOU CAN ADD AN EXTERNAL SWITCH (JACK) – When a ROMPA® switch is plugged into the Jack Socket on the side of Push Button Switch Box, most other controls will be overridden, i.e. Push Buttons, Colours etc.



There are nine possible options when a ROMPA® switch is used. Use the dials on the underside of the Super Interactive Switch Box:

1. With the left hand dial set to COLOURS and the right hand one to FLASH ON, then the Bubbles will be on all of the time and the Lights will come on when the ROMPA® switch is activated.
2. With the left hand dial set to BUBBLES and the right hand one to FLASH ON, then the Lights will be on all of the time and the Bubbles will come on when the ROMPA® switch is activated.
3. With the left hand dial set to COLOURS and the right hand one to FLASH OFF, then the Bubbles and Lights will be on all of the time and the lights will stop when the ROMPA® switch is activated.
4. With the left hand dial set to BUBBLES and the right hand one to FLASH OFF, then the Bubbles and Lights will be on all of the time and the Bubbles will stop when the ROMPA® switch is activated.
5. With the left hand dial set to COLOURS and the right hand one to HOLD, then the Bubbles will be on all of the time and the Lights will start when the ROMPA® switch is activated and will only stop when the ROMPA® switch is activated again.
6. With the left hand dial set to BUBBLES and the right hand one to HOLD, then the Lights will be on all of the time and the Bubbles will start when the ROMPA® switch is activated and will only stop when the ROMPA® switch is activated again.



7. With the left hand dial set to BOTH and the right hand one to FLASH ON, then the Bubbles and Lights will be off and they will only come on when the ROMPA® switch is activated.

8. With the left hand dial set to BOTH and the right hand one to FLASH OFF, then the Bubbles and Lights will be on all of the time and the Lights and Bubbles will stop when the ROMPA® switch is activated.

9. With the left hand dial set to BOTH and the right hand one to HOLD, then the Lights and Bubbles will be on all of the time and both will stop when the ROMPA® switch is activated and will only start when the ROMPA® switch is activated again.

14. Care and Maintenance

- Wipe clean with a damp cloth.
- Dry thoroughly immediately after wiping.

15. Troubleshooting

Not applicable.

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at producthelp@rompa.com