1. Product Name

Snoezelen® Snug

2. Product Code

19272

3. Colour

As above

4. Brief Description

The Snoezelen® Snug provides a multi-sensory environment designed for exploration, stimulation and relaxation. Easy to assemble and ideal for those who have limited space. Can be disassembled for storage.

5. Contents

- 1 x padded floor
- 1 x roof
- 4 x walls with Tac-tiles already attached

6. Snoezelen® Stimulations

- Touch
- Sight
- Sound

7. Best Use

A portable piece of equipment that can be assembled and disassembled for storage.

8. Compatible ROMPA® Products

ROMPA® offer a range of practical, rehabilitation, healthcare and sensory resources for people who have a range of needs. Please look at www.rompa.com for our full range.

9. Starting Up

- Choose an appropriate and safe location for the activity

10. Detailed Description

Provides a multi-sensory environment designed for exploration, stimulation and relaxation. Easy to assemble and ideal for those who have limited space. Can be disassembled for storage. Equipped to suit every possible situation. The sensory hut includes a padded floor, four walls, and a PVC roof. The walls, both inside and out, feature a different tactile sensation.

11. Safety

Always supervise the use of this product.

- Do not stand on the roof of the Supa Sensory Hut – it is not intended to be climbed on.

12. Technical Specification



Size: 145W x 145L x 165Hcm

13. Installation

Two people are needed to assemble this product.

Secure the walls around the floor using the attached Velcro®. It is easiest to put two walls together first then push the floor mat in place and finally attach the two last walls. There is no wrong combination for attaching the walls but it is recommended that the doorway is opposite the mirror and the Tac-tiles are on each side. For aesthetic reasons, you may wish to make sure the colour of the Velcro flaps matches the colour of the wall they are attached to e.g. green flap to green wall. Once all four walls are in place, attach the roof, again using the Velcro. The cream side of the walls should be inwards.

Remove the protective film off the mirror before use.

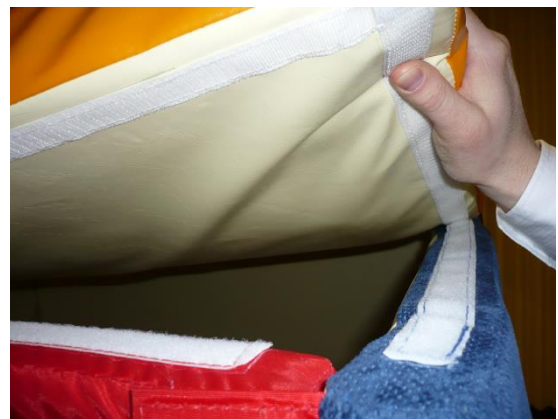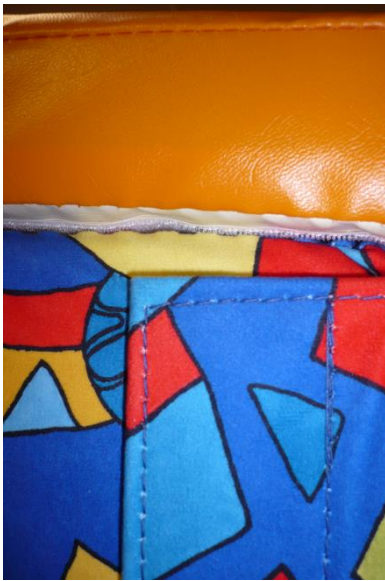14. Care and Maintenance

Ideally store flat. Do not place heavy items on the stored product – this may put too much pressure on the wood interior and make future assembly difficult.

ROMPA® Soft Cushioning is made from high quality durable vinyl, PU Nylon or Soft & Silky with a high grade of fire retardant soft foam. ROMPA® Soft Cushioning exceeds all European Standards for fire retardancy for the combined use of vinyl/PU Nylon/Soft & Silky and plastic foam. If you look after your soft cushioning it should give you many years of service.

Nonetheless, please observe the following:

1. Vinyl will crack, tear and “rub off” if subjected to sharp objects or continual rubbing by hard objects.
Avoid shoes, buckles or sharp/hard objects.
2. Organize a regular cleaning schedule.
3. Wipe off accidental marks/liquids using a mild detergent and a damp cloth, avoiding excessive amounts of water. Do not use cleaners with abrasives in them or solvent-based cleaners such as trichloroethane or carbon tetrachloride. Liquid detergents used to wash cups/plates etc. are ideal. Do not steam clean.
4. From time to time lift floor cushions to clean all surfaces and the floor underneath.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at

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