



EMPATHY CAT 19924



CONTENTS

1 x Cat

BEFORE USE

The cat may be used just as he is; as a puppet or warmed before use.

1. To warm the cat up, open the cat up between his back paws and remove the white inner cushion.
2. Place the white cushion in a suitable jar/container and place the jar/container in a larger bowl/pot containing boiled water. Leave the jar to warm through for no more than ten minutes. Do not allow the boiled water to come into contact with the white cushion.
3. Place the white cushion back inside the cat and close the cat back up.
4. Make sure the cat is a safe and comfortable temperature before passing him to your client.
5. Do not use a microwave or oven to heat the white cushion.
6. Only heat the white cushion, not the cat itself.
7. Handle the warmed white cushion with care.

SAFETY

1. Always supervise the use of this product.
2. Always check the temperature of the warmed cat before handing it to a client.
3. Take care not to scold anyone with the boiled water. Keep the water out of reach.

CARE AND MAINTENANCE

1. Remove the inner white cushion and hand wash the cat exterior with care in tepid water.
2. Allow to air dry naturally.
3. Do not bleach; do not tumble dry.

SPECIFICATION

Size: 30 x 18 x 30cm

Weight: 1.1kg