TALKING TIN – RED 20362


CONTENTS

- 1 x Talking Tin – Red (with batteries already installed)


BEFORE USE

To record a message

1. Pull the tab on the rear of the 'Tin' to activate the battery power.
2. Hold the 'Tin' in both hands with your thumbs over the two record buttons.

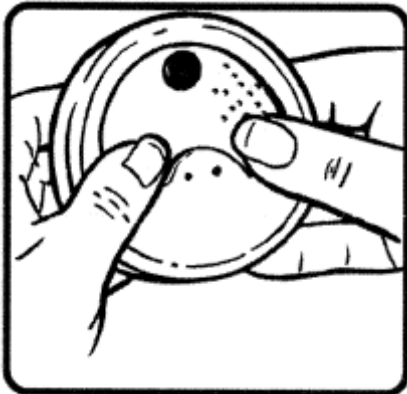
3. Press down with your thumbs and the red light will illuminate to indicate that recording has begun. Make sure that both your thumbs remain pressed down so that the red light remains on whilst you are recording. Speak clearly at a distance of approximately 15cm (6") and release your thumbs when you have finished recording.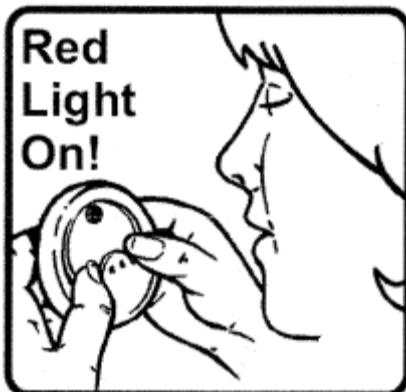

This product will allow you to record one single 40 second message. You can re-record as many times as you wish. New recorded messages will override previous messages.

If you wish to protect the message you have recorded, slide the switch on the back to the LOCK position.

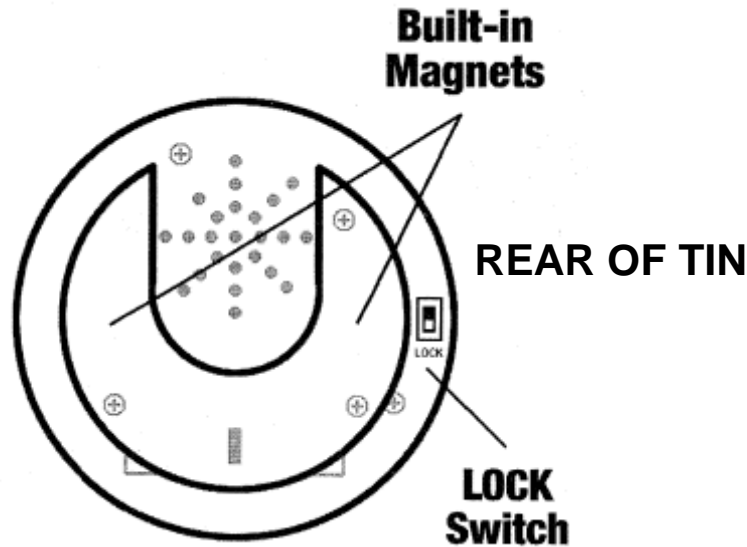The LOCK switch stops the recording function from being activated. Remember to slide the switch back again when do eventually wish to change the message.

OPERATION

To play a message

1. Press the PLAY button once to hear your message.

2. There is NOT a volume control.

The magnets in the product allow the Talking Tin to be attached to a can or tin. This can be a practical aid for verbally 'labelling' the contents of tins for the visually impaired.

There is a 2.4cm wide and 3mm high gap between the U section at the rear of the product and the 'Tin' itself. This allows a thin belt or a piece of elastic to be passed through the Talking Tin to allow the product to be carried. It also allows the Talking Tin to be tied to an object e.g. a bottle.


SAFETY

Always supervise the use of this product.


CARE AND MAINTENANCE

To replace Batteries


1. Each Talking Tin requires three AG13 (LR44) button cell batteries. With the rear of the 'Tin' uppermost, remove the U shaped section by loosening the three screws.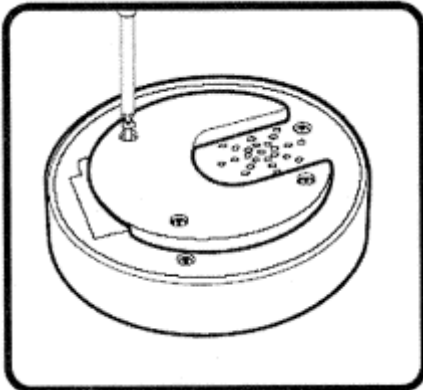

2. Remove the screw holding the battery compartment cover.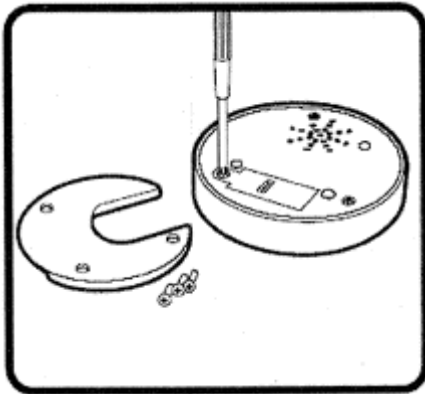

3. Replace all three batteries, observing the polarity required.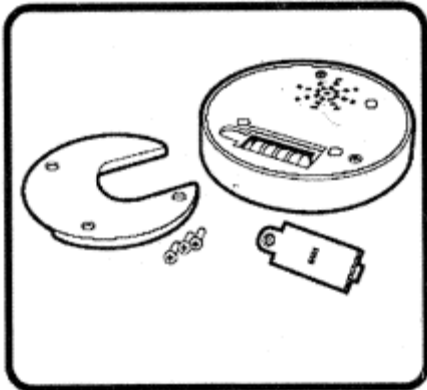

4. Replace the battery compartment cover and screws.
5. Use Alkaline batteries only.
6. Do not mix old and new batteries.


To clean

Wipe with a soft, barely moistened cloth. Avoid abrasive and harsh cleaning products as these will scratch the plastic.


Keep dry.

Once this product is eventually exhausted, please dispose of it appropriately.

SPECIFICATION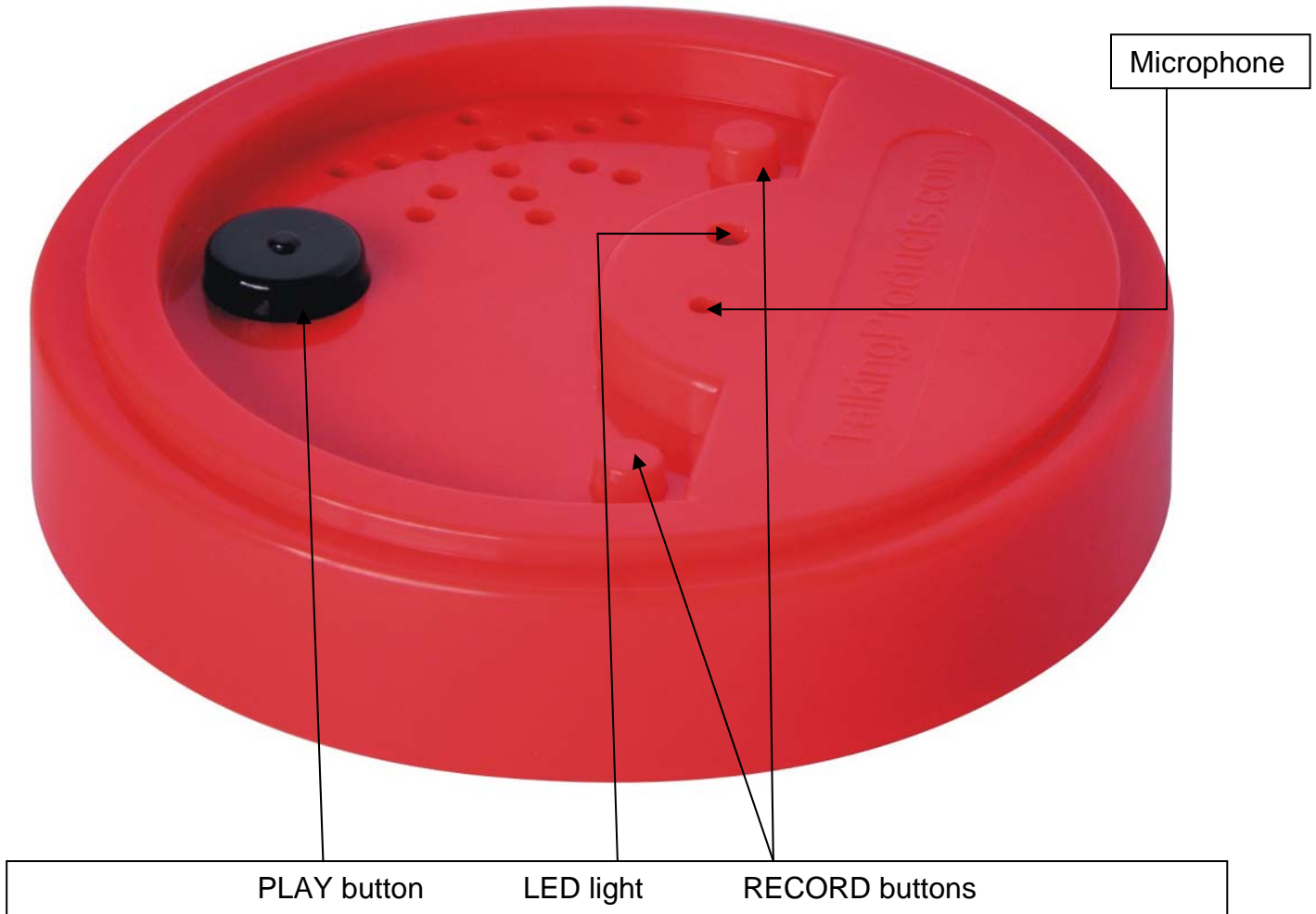

REAR OF TIN

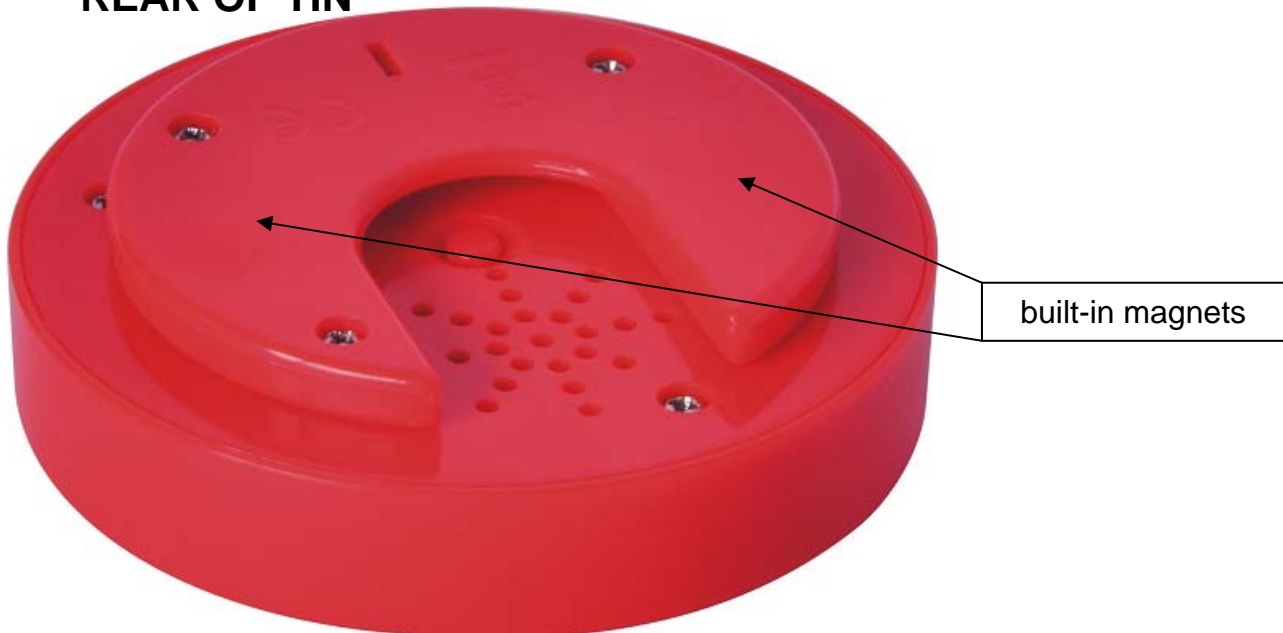Further copies of these instructions can be downloaded at www.rompa.com
We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at producthelp@rompa.com

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