



1. Product Name

Gondola Bean Bag

2. Product Code

20445

3. Colour

Typically as shown.

4. Brief Description

A tactile soft seating area which can be used by adults or children for sitting.

5. Contents

- 1 x Gondola Bean Bag

6. Snoezelen® Stimulations

- Tactile

7. Best Use

Use as seating for children or adults

8. Compatible ROMPA® Products

Many multisensory items can be used as passive or interactively whilst the client/s are sitting on the Gondola Bean Bag.

If you wish to increase sensory stimulation for your client/s, please refer to our catalogue or website www.rompa.com for a great selection of items that provide a huge variety of olfactory, tactile and visual input.

9. Starting Up

- Unpack item
- Decide where it is to be placed, considering potential risks and hazards in the surrounding space.
- Place on an even floor.
- Decide how the seating area will be used and position appropriately to support the user.
- Help user transfer safely into an appropriate position for use (if appropriate).
- Ensure client is supervised using this product at all times.

10. Detailed Description

This gondola is suitable for most children and adults. It is intended to be used as a seating item to sit comfortably with some support for the back and neck whilst engaging in other activities.

11. Safety

- For indoor use only.
- For daytime use – it is not intended to be slept upon.
- For 3 years and over.
- The Gondola Bean Bag should not be mouthed.
- Remove all packaging before use and store or dispose of this appropriately.
- Keep and use away from sources of heat e.g. fire, radiators etc



Please read all the Safety information below before each use

Remove all packaging and either store this in a safe place out of reach of clients or dispose of this responsibly. Plastic bags and wrapping can be hazardous to children and vulnerable people.

This product contains a bead filling. The beads in the bead filling are small parts and should always remain out of reach. Do not attempt to access the interior part of this product. Never allow children or vulnerable people to access the interior part of this product.

Should any seams or the zip become damaged, contact ROMPA® immediately to arrange repair or replacement. Should the product require re-filling, contact ROMPA® immediately to discuss this.

Keep away from sources of heat and fire.

ALWAYS SUPERVISE THE USE OF THIS PRODUCT

12. Technical Specification

- Material: 100% cotton outer
- Filling: polystyrene
- Size: 110 x 100cm

13. Installation

- Unpack the product and dispose carefully of packaging materials.
- Use the seating area according to client need. Provide assistance if required to transfer to use the item.
- Please ensure that when positioning the Gondola Bean Bag in a space that a risk assessment of the surrounding space has been carried out to ensure the safety of the user and other people. Ensure that walkways and doors are not obstructed and that the item is not positioned near to sharp edges such as furniture corners etc to minimize potential risk if client/s move around.

14. Care and Maintenance

- Keep away from sharp and pointed objects.
- The Gondola Bean Bag is intended to be sat/laid on – not jumped on.
- Regularly check the Gondola Bean Bag for wear and tear. Discontinue use if damaged until repaired or replaced.

Observe the following:-

1. The outer is washable is 40°C.

2. Do not bleach.



3. The outer may be tumble-dried at a low heat.



FABRIC MAINTENANCE

If you look after your Gondola Bean Bag it should give you many years of service, but please observe the following:-

1. Fabric may wear, tear or “rub off” if subjected to sharp objects or continual rubbing by hard objects. Avoid shoes, buckles or sharp/hard objects.
2. Organise a regular cleaning schedule.
3. Wipe off accidental marks/liquids using a mild detergent and a damp cloth, avoiding excessive amounts of water. Do not use cleaners with abrasives in them or solvent-based cleaners such as trichloroethane or carbon tetrachloride. Liquid detergents used to wash cups/plates etc. are ideal.
4. From time to time lift the seating area to clean all surfaces and the floor underneath

15 Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at producthelp@rompa.com

SACO DE FEIJÕES GÔNDOLA 20445



CONTEÚDO

1 x Saco de feijões Gôndola

SEGURANÇA

Supervisione sempre a utilização deste produto

CUIDADO E MANUTENÇÃO

Limpeza

1. O exterior é lavável a 40°C

2. Não utilize lixívia.



3. O exterior pode ser seco na máquina a baixas temperaturas.



ESPECIFICAÇÕES

100% algodão

Enchimento: poliestireno

Tamanho: 110 x 100cm