



1. Product Name

Interactive Aroma Panel by ROMPA®

2. Product Code

20943

3. Colour

Frosted opal finish

4. Brief Description

Activate smells, colours, sounds and breezes for aromatic, visual, audible and tactile stimulation. LED lights indicate which aroma is being diffused.

5. Contents

- 1 x panel
- 1 x complimentary pack of sample aromas

6. Snoezelen® Stimulations

- Sight
- Sound
- Smell

7. Best Use

Wall mount indoors and use as a stand-alone product or with a Wi Fi controller or Sensory Magic™.

8. Compatible ROMPA® Products

20848, 21098 – Sensory Magic™

19847 - Deluxe 8 Colour Wirefree Controller

21006 - 8 Colour Wirefree Controller

19939 – Wi Fi Interactive Carpet switch Set by ROMPA®

21728 - ROMPA® Wi Fi App Control Box

18870 - Talking Cube by ROMPA®

20402 - Mini Talking Cube

21917 - Wi Fi Microphone by ROMPA®

22716 - ROMPA® Wi Fi Colour Control Bumpas - Set of 8

21918 - Wi Fi Override Transmitter

22867 - Wireless Colour Controller Wall Mat

22871 - Colour Command Centre™

9. Starting Up

- Wall-mount.
- Fill aroma cartridges.
- Plug in and activate the panel.

10. <u>Detailed Description</u>

Activate smells, colours, sounds and breezes for aromatic, visual, audible and tactile stimulation. LED lights indicate which aroma is being diffused. Compatible with Sensory Magic™ and our ever-improving range of Wi Fi controllers, this panel can be controlled easily and remotely. Includes 4 aromas (more available separately). The integral fans have 2 speed settings. To be wall-mounted (horizontally or vertically). Frosted opal finish only.

Using the Interactive Aroma Panel:

- 1. Plug the Panel into the mains.
- 2. Turn the Panel on using the On/Off switch on the left-hand side of the Panel. The four shaped buttons will then light up in four different colours.
- 3. Press one of the four shaped buttons and be rewarded by sound, colour, breeze and aroma. A spraying sound is played as the aroma is blown out by the internal fan. When pressed, the button colour changes to green. The button stays green and the internal fan remains on for the duration of the button press: Press and hold the button for as long as desired.
- 4. The button will revert back to its original colour if not pressed.

WiFi Operation:

The Interactive Aroma Panel may be used with other ROMPA® compatible Wi Fi products: Simply press any ROMPA® Wi Fi switch, and the Interactive Aroma Panel buttons will change to the colour selected. The Wi Fi colour will remain for one minute, unless another Wi Fi colour is chosen. The Interactive Aroma Panel buttons then revert back to their original colours.

Sensory Magic[™]:

The Interactive Aroma Panel is compatible with Sensory Magic 2TM and may be controlled from within the Sensory MagicTM system.

11. Safety

• Supervise the use of this product.

12. Technical Specification


Size: 60 x 25 x 10.5cm Weight: approx 12kg

Power: 12v 60W (12v x 5A)

13. Installation


Wall-mounting

Use the pre-drilled holes (one at each of the corners) to wall-mount the panel. No fixings are included in this product for wall-mounting. When fixing it to the wall please consult a suitably qualified professional builder or from an estates department to understand the construction of the wall the panel is to be fixed to, so that the correct fittings can be used.

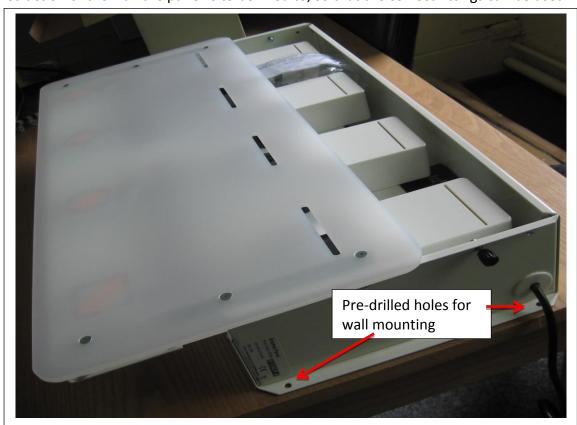

Filling cartridges with aroma

1. Aromas can be added to each of the 4 aroma cartridges inside the Interactive Aroma Panel. Access the cartridges by unlocking the front of the Interactive Aroma Panel.

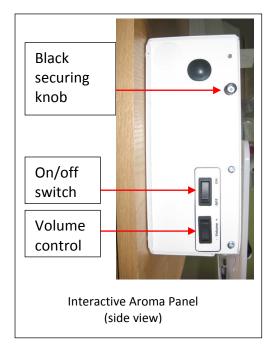




- 2. Unscrew the two black securing knobs. The Panel lid will then be able to slide open.
- 3. The four cartridges can then be seen inside the Interactive Aroma Panel.



Aroma cartridge in position

- 4. The four Aroma cartridges are held in place with magnets. Simply pull them off to remove.
- 5. To fill a cartridge with Aroma, either remove all four screws on the back lid, or, remove the bung/vent on the underside:



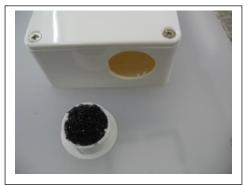

6. Cartridges can be filled with the aroma of your choice: use scented beads, sachets, or scented oils. Fragrance or aromatherapy essential oils can be applied to cosmetic pads / small sponges and placed inside the cartridges.

Please note: essential oils are strong, concentrated and potent, hence only a few drops are required. Use essential oil sparingly, and make sure that the pad/sponge etc. is not saturated. Do not allow the oil to come into contact with the plastic cartridges (place the pad/sponge in a small ball of cotton wool for example). Certain oils, especially citrus oils (orange, lemon), cinnamon, oregano and thyme are especially caustic towards plastic.





- 7. Refit the cartridge back lid by replacing the four screws *OR* Refit the cartridge bung/vent.
- 8. Place the filled Aroma cartridge back inside the Interactive Aroma Panel. Aroma cartridges are numbered 1-4. Simply line up the numbered magnet on each cartridge with the corresponding number inside the Panel:



9. Slide back the Panel lid and retighten the black securing knobs.

14. Care and Maintenance

- Wipe clean with a soft, very slightly dampened cloth.
- Do not use abrasive cleaning products or bleach.

15. <u>Troubleshooting</u>

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at producthelp@rompa.com