


Doo-zzy™

doo-zzy/'doozē/ - Noun: **Something outstanding or unique of its kind**

User Manual

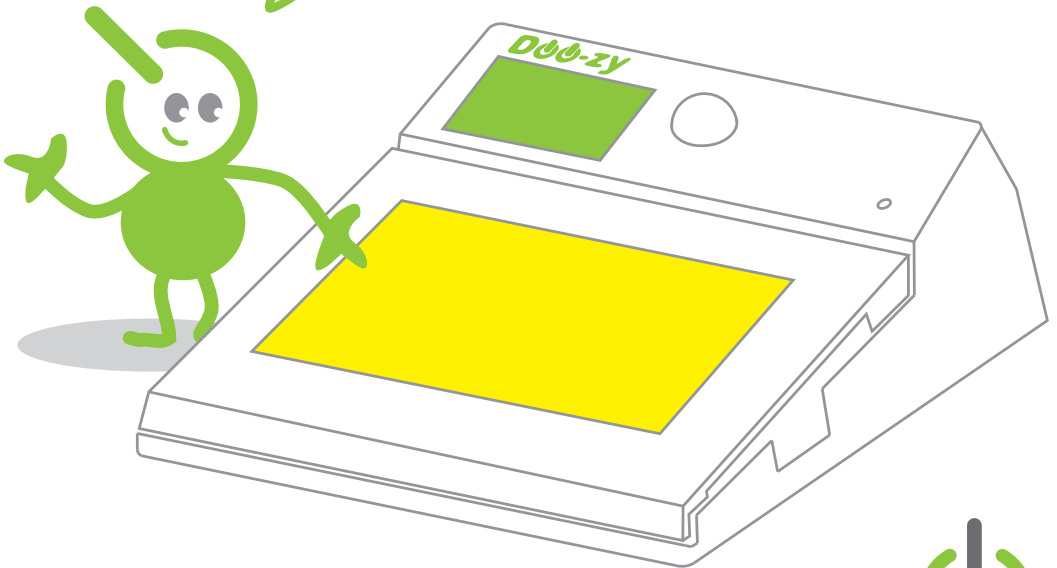

Doo-zzy manual Ver 1A
Pat Pending 1016779-9



Register your Doozy at

doo-zzy.co.uk

Copyright Switch-Ed 2011

Index

Hello!
The Doo-zy
is easy to use. This
manual will guide
you through.


Quick Start
Jump to page 10

Introduction

List of contents	1
Introduction	2
Unique features	3
Doo-zy tour	4
How it works	5
Batteries and charging	6
Select Button	7
Programs and functions	7
Select button and function menu	8
LOC - Special function	9

Using Doo-zy

Helpful information	9
Turning On and Off	10
Record sound and playback with inbuilt mic	10
Volume control	11
Switch output programs	11
Eye function	12
Vibration function	12
Display function	13
Switch pad illumination function	13
Record with MP3 player or external microphone	14
Infrared remote control learner	15
Wireless infrared function	16
Default and reset function	16

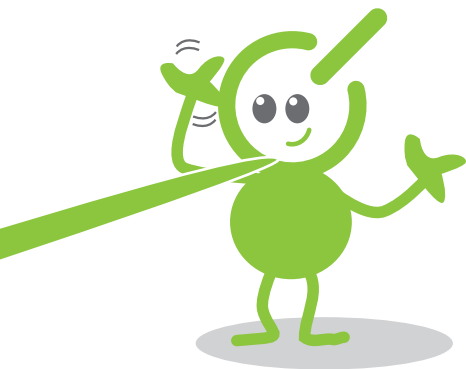

Warnings

Doo-zy should always be used under supervision and especially with children and users with special needs!

Doo-zy is designed for use indoors only and must not be used in wet and humid areas.

Switch Output & Programs

How programs affect the output	17
Direct output	18
Timed output	19
Latch output	20
Sequence output	21
Opposite output	23
Timed Out output	24

Attachments & Mounting


Positioning your switch	25
Fitting low profile pad	25
Graphic card slot	26
Cradle switch bracket	26
Cradle for table top	27
Mounting point for adjustable arm	27
Replacing the batteries and charging	28

Technical

Technical specification	29
Factory defaults	29
Trouble shooting & hard reset	30
Compliance and legals	31
Consumer Warranty	32

List of Contents

Thank you for purchasing the Switch-Ed Doo-zy. Please check the contents of the package and ensure you have all the parts listed below. Your switch comes with a one year warranty from date of purchase. A copy of our guarantee and warranty is listed at the rear of this booklet.

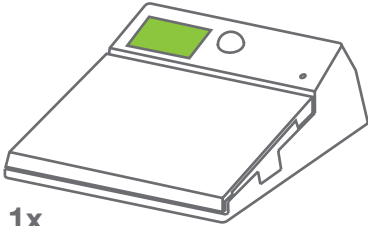


1x

Doo-zy




1x

Quick Start guide

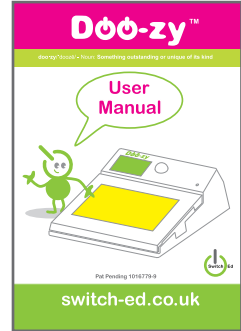


1x

User Manual

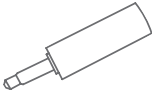


1x

3-5 mm jack adaptor

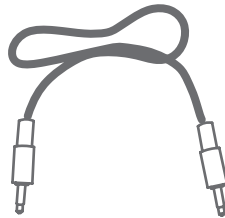


1x

Lead

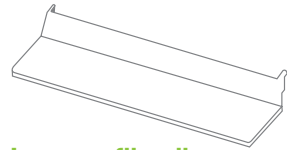


Low profile clip

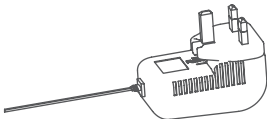


1x

Power supply charger

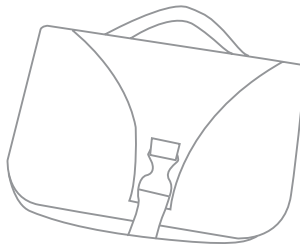

1x

Carry bag

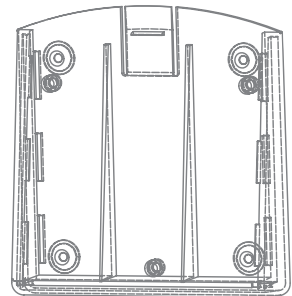

1x

Cradle bracket



10x

Graphics card

This manual is designed using lots of friendly graphics and symbols to help you learn how to understand and operate the switch as quickly as possible. We suggest you have the switch next to you together with this manual and work through whilst quickly cross referencing when required.

Introduction

What is the Doo-zy Switch?

Doo-zy Switch is a unique device designed for people with a wide range of special needs. It is a recordable switching communicator that can be used as a standalone device or with toys or assistive technology control systems such as automatic doors and curtain openers.

Doo-zy Switch combines a number of devices already available in the market place and builds them in to one package along with some very unique features.

The case is robust and constructed from extremely strong glass filled ABS plastic. It uses micro electronics and is virtually a small computer using firmware developed exclusively by Switch Ed.

Doo-zy is extremely flexible and easy to use.

Doo-zy can be used as a :-

- Recordable Talking Switch
- Step communicator
- Infrared universal remote control
- Infrared transmitter
- Music player
- Magic Eye switch
- Count up, count down switch with counting display
- Switch controller for toys with built in direct/momentary, latch, opposite, timed, timed off and sequence programs.

Wow!
8 features
in 1



External devices and toys


Doo-zy allows you to connect an external switch fitted with a 3.5 mm jack plug so functions and programs can be used with a favourite switch.

A reward device such as a toy can also be connected using a 3.5 mm jack lead.

We can also program the Doo-zy as a simple universal remote control. It can be trained to control infrared devices such as toys or even your television.

Unique Features

Its unique features includes a **superior audio sound quality** of 12 khz. This is immediately evident when recordings are played back. An on board micro SD memory card is capable of storing hours of recorded sound. Record with the in built microphone or record your favourite music from any audio source including MP3 players.

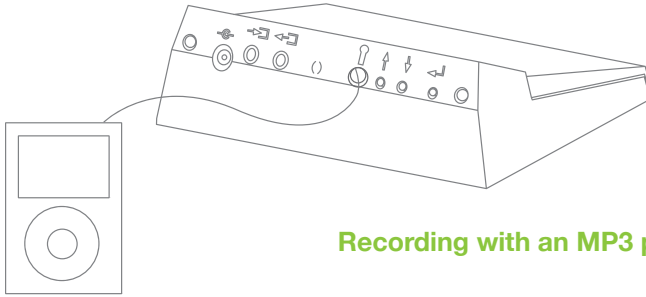

Recording with an MP3 player

A **transparent switch pad with a graphics card slot** allows you to make your own symbols and graphics quickly and easily. An added bonus is that it illuminates making it a great visual reference point.

An on board **Magic Eye** uses infrared technology that detects human heat which allows users to operate the switch without touching the switch pad. In other words an invisible beam with two ranges short and long.

A **vibration device** gives vibratory and tactile feedback when the switch pad is pressed.

Program and function menus are selected using easy to use up, down and select buttons. The LED **bright 7 segment display** shows which program or functions are running at any one time.


All functions are also de-selectable and can be switched off if they are a distraction to users.

On board **infrared receivers** and transmitters allow Doo-zy to learn infrared codes from household equipment such as TVs . This effectively turns Doo-zy into a remote control to operate a wide range of equipment. Optional IR receiver gives wire free operation with non infrared equipment if required.


The use of surface low power micro technology allows the **rechargeable batteries** to last for a long period of time whilst an overnight charge completely replenishes the batteries. Doo-zy will run for weeks in standby mode.

Doozy Tour

Front view

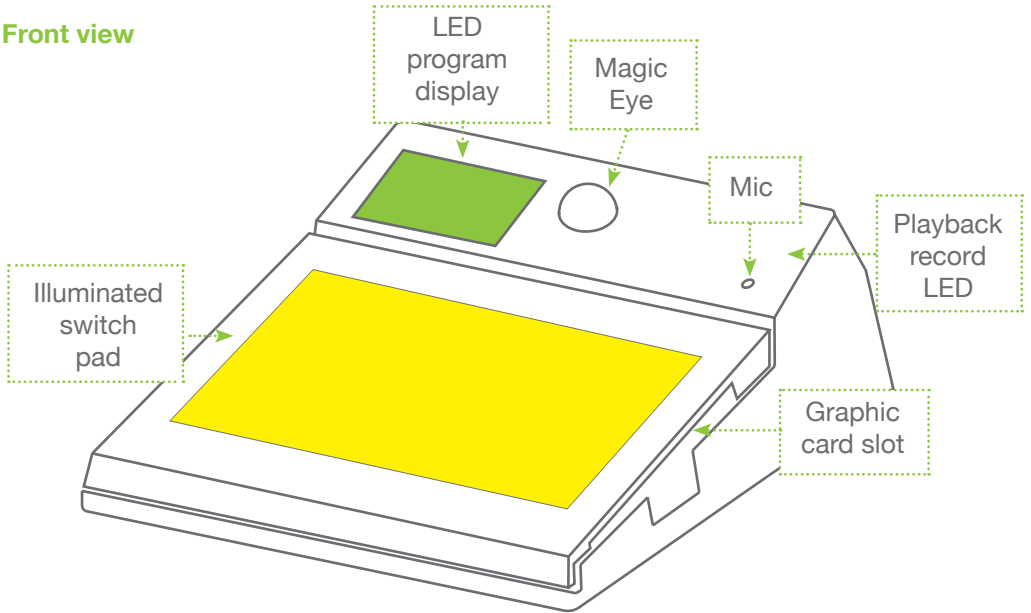


Rear view

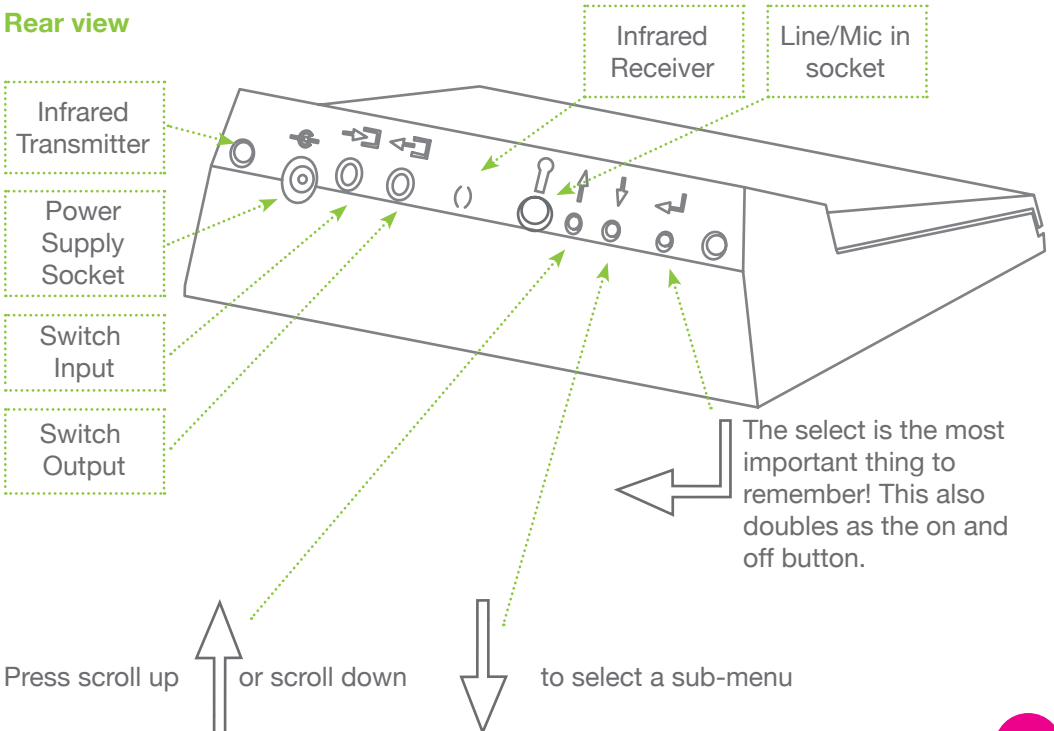


Doo-zy & how it works

Doo-zy uses micro technology to operate external devices but how the switch pad reacts and interacts with the users and devices depends on how the built in programs and functions are set.


Below is a schematic that may make it a little easier to understand.

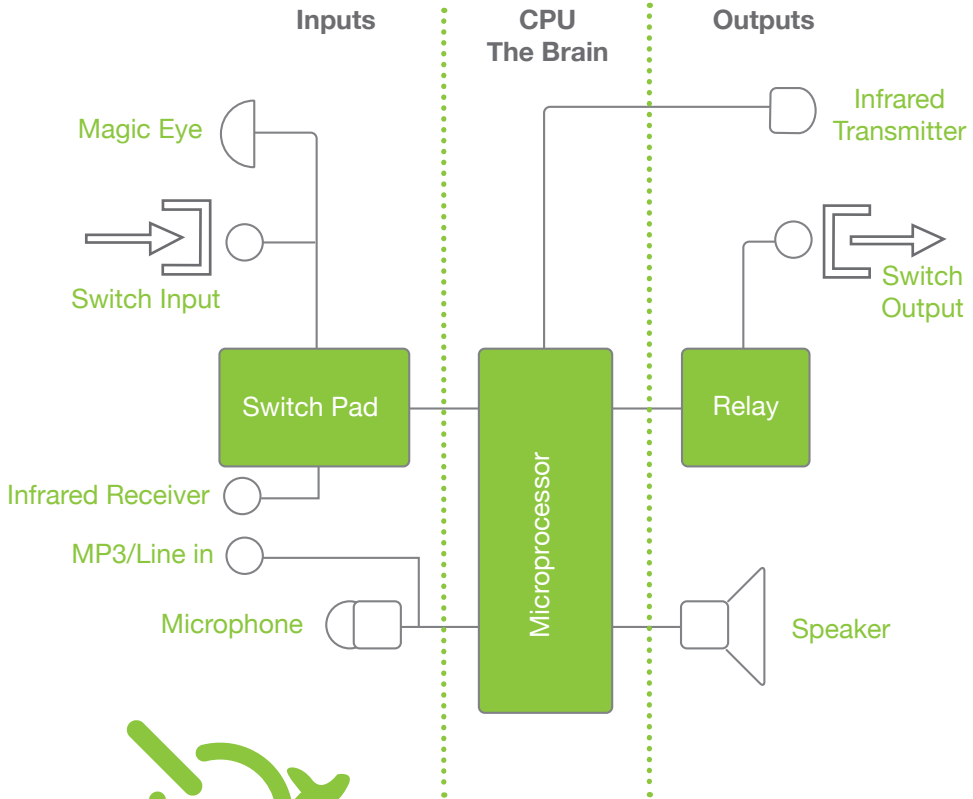

Batteries & charging

Doo-zy is supplied fitted with a special rechargeable battery pack. An operation time of around 6.5 hours constant use should be possible depending which features are being used. If, for example the vibration is used constantly this would reduce battery operation time. Doo-zy has an on board low power battery indicator, when Bat  is shown in the display they need recharging.

Battery Life


We suggest that batteries should only be charged when the Bat indicator is displayed and then fully charged for 5-6 hour period. This will ensure maximum battery longevity and should be good for around 500-800 charges. When  displays constantly it is time to replace batteries.

Battery Replacement – See page 28.

Charging


Doo-zy is supplied with a world wide power supply charger. It can be used with a range of voltages from 110-240V. A range of four plugs are supplied for different areas of the world. USA/North America, Europe, UK and Australia.


Choose the correct adapter for your country and simply slide on the power supply until a click can be heard. To remove push the indicated pressure point and remove

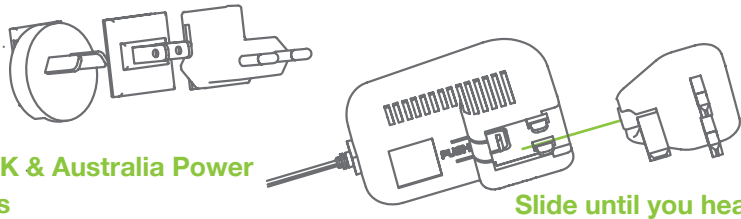

USA, Europe, UK & Australia Power Supply adapters

We suggest a full charge of at least 5 hours when you first unpack your Doo-zy.


Insert the small plug on the power supply into the rear of the Doo-zy marked with this symbol  and plug into the mains power supply.

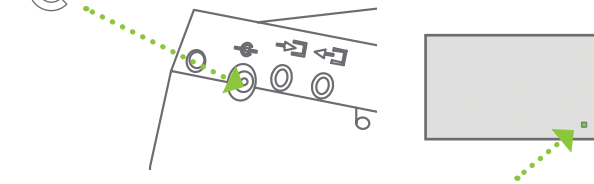

When the battery is charging a flashing decimal point indicates it is receiving power from the charger. This is a good way to check the power supply is working correctly. This only flashes when Doo-zy is turned on.

Note: Under no circumstances use any other charger, permanent damage may occur.

Select button

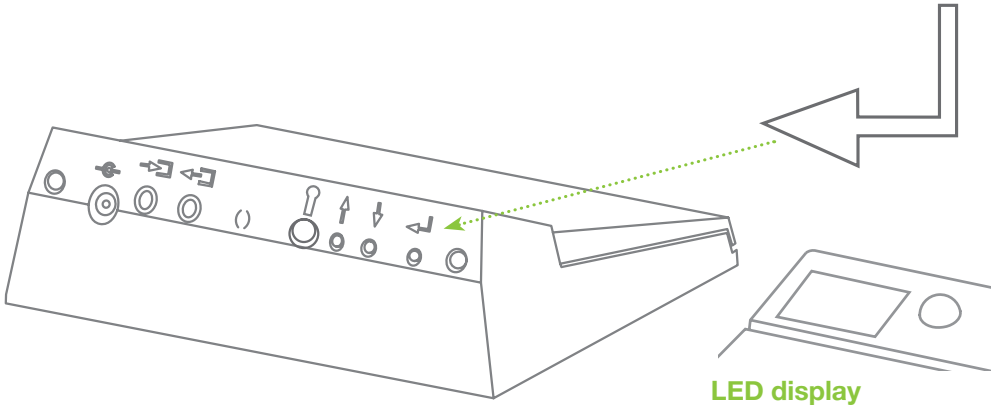

The select button is the most important button to remember as it allows access to almost all of the functions and sub menus of Doo-zy!

When using the select button the LED display will inform you which menu you are in.


Switch Output Programs and functions

The Doo-zy uses internal pre-set **programs** to make the switch output perform complex or simple tasks that involve the recorded sound.


More information on the switch output programs on page 17.

Doo-zy has a range of features called **functions** that can be enabled or disabled. Some functions could be distracting to some users. A good example may be the vibration and LED display. All can be disabled in the menus.

More information on function page 8.

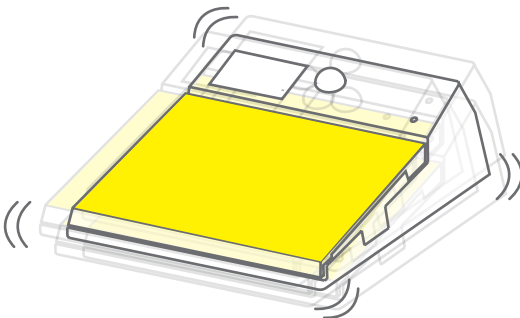

Vibration & LIT function on.

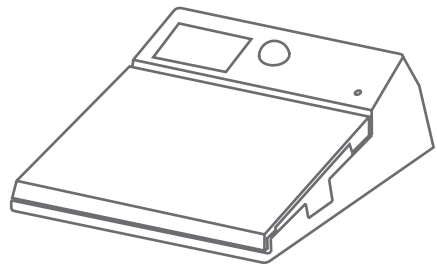





Vibration & LIT function off.



Select button & function menu



Every time the select button  is pressed a new menu is displayed



Page guide



 **x1**  **REC** - This is the selection we make to record a sound **Page 10 & 14**

 **x2**  **VOL** - Speaker volume adjustment **Page 11**



 **x3**  **OUT** - This option decides how the Doozy output reacts when the switch pad is pressed **Page 17**



 **x4**  **EYE** - This gives an option of having the Magic Eye off and the beam near or far **Page 12**

 **x5**  **VIB (Vibration)** - This gives an option of vibration on, off or pulse **Page 12**

 **x6**  **DIS (Display)** - This gives an option of turning the LED display off, up or down **Page 13**


 **x7**  **LIT (Switch pad illumination)** - This gives an option of Switch pad illumination on, off or pulse when pressed **Page 13**


 **x8**  **IR (Infra Red)** - This is the menu for all Infra red options, including learning **Page 15 & 16**

 **x9**  **DEF (Default)** - This gives access to the factory defaults **Page 16**


Note - Each menu is displayed for 5 seconds only

LOC Special function

We have a special function that is not chosen with the select button, this is the  LOC function. This locks all of the three buttons that are used at the rear of the Doo-zy.

When locked none of the buttons will operate and LOC  will be indicated in the display.


To enable the LOC function press and hold the middle button (arrow down ) for more than two seconds. LOC  will be displayed.

To unlock the function repeat the process and after two seconds UNL  will be displayed.

Using Doo-zy

Helpful information

Doo-zy can perform all kinds of tasks you will find unbelievable. However, we suggest you follow a simple procedure every time you are preparing a new task.

Step 1:- Always choose your switch **output** program . This way you will learn to understand how Doo-zy works. See page 17 for Output info.




Step 2 :- Choose your **functions**, example, vibration on, switch pad lit. See page 8 for function menu.


Step 3 :- Record your sound if required.

We strongly suggest you register your Doo-zy so we can keep you informed of any firmware issues and updates by going to our web site www.doo-zy.co.uk


Using Doo-zy


Turning on and off






Press the select button  and hold for three seconds. A decimal point  will scroll across the display and then  will be displayed, it is now ready to use.

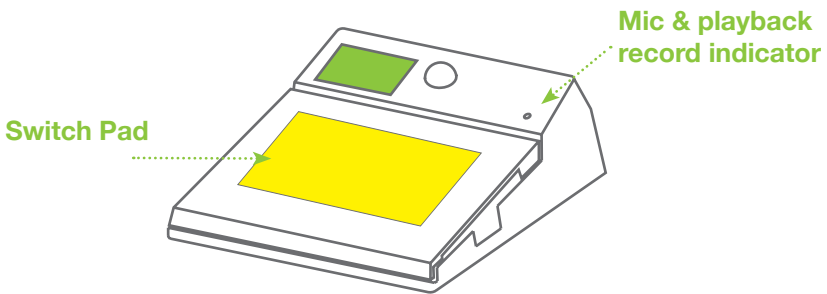If you want to turn off the switch at any time press and hold the select for more than 3 seconds until  shows in the display.

Record sound & playing back with in built microphone



We are now going to use the onboard microphone to record a sound. If you have not done so already switch on the switch by pressing and holding the select button  for more than three seconds.






Press the select button  until  shows in the display, now push down the switch pad  and hold . Now talk clearly at about 20 cm away from the mic  on the front panel and anything you say will be recorded. Now release the pad. Press the switch pad  again to listen to your recording.

Note: The playback / record LED glows red  when recording and green  when playing back. Anytime the pad is now pressed the recording will play back.

We can record directly from an MP3 player using a cable. More about this on page 14


If the volume is too loud jump to page 11 where we will show you how to alter it.


Volume control

The volume of the recorded message can be altered by pressing the select button  until the display shows VOL  Use the up / down arrow buttons   and choose a number between 1 and 10 with 10 being the loudest and 1 the lowest volume. Press the select button to confirm your choice and SET  will be shown in the display.



Switch output programs



Note: Switch output is used to connect external devices such as toys.


The Doo-zy uses internal pre-set programs to make the switch output perform complex or simple tasks that involve the recorded sound. These are changed in the OUT  menu. For more information go to page 17.

Lets have a look at the OUT  menu.

Press select button  until  shows in the Display.

Now to view all the options in the out menu, just press either of the scroll arrows  

Stop on the program you wish to use and press select  to confirm. SET  will show in the display.

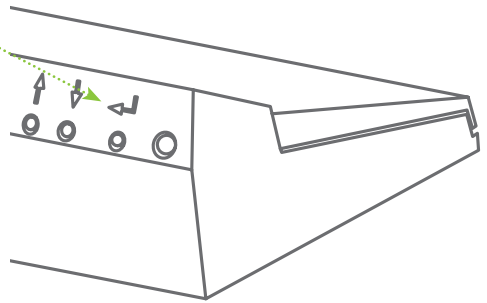

Note: When in a sub menu the select button acts as a confirm button.
Each menu is displayed for 5 seconds only.

Other programs listed in this menu are listed below shown with the pages where you can find more detailed information.



Direct - Page 18



Timed - Page 19



Latch - Page 20



Sequence - Page 21





Opposite - Page 23



Timed out- Page 24



Eye function

The Eye uses an invisible beam so users do not have to press the switch pad. It uses an infrared detector to measure light radiated from a person therefore it is very sensitive.

The switch pad is still active whilst the eye is on. The eye has two settings for the beam length near  and far . It is factory set to off.

To turn on the eye press the select button  until the display shows 

Use the arrow buttons   to select near  for a short beam length of around 1 meter or the  for a far beam length of around 2.5 meters.

Press the select button  to confirm and SET  will show in the display.




Note: On initial start up the eye takes a few seconds to self test the internal electronics it also waits 5 seconds between triggers to stop repetition.





Note: The eye will not operate the Doo-zy when in Direct, Latch and Opposite mode.

Vibration function

Vibration is used to give tactile feedback when the switch pad is pressed, it pulses to give maximum effect.

We have three options in the Vibration menu:

- Pulse  This option pulses the vibration every 4 seconds without any switch pad activation.
- On  The vibration activates only when the switch pad is pressed.
- Off  The vibration is always off.



Use the arrows   to make your choice and press the select button  to confirm and SET  will show in the display,

Display function

Display refers to the seven segment LED display on the front panel. It is factory set to count down in seconds unless disabled in the DIS menu.

To disable the display press the select button  until the display shows DIS 

Use the arrow buttons   to select off 

Press the select button  to confirm and SET  will show in the display.

Press the switch pad and the display is now OFF.


Other options available is count UP  . The display counts up from zero counting in seconds. The other option is count DOWN  timer counting in seconds.





Note :- The display will display minutes and seconds when recording long sounds.


Switch pad illumination function


The switch pad illuminates when the pad is pressed to give strong visual feedback, it also pulses every five seconds.

The options in the LIT  menu are:

-Pad  Choose this to enable the illumination function of the switch pad when pressed.

Use the arrows   to make your choice and press the select button  to confirm and SET  will show in the display,

-Off  Choose this option to turn off all switch pad illumination.

-Pad  Choose this option to have the pad illuminate with a pulse light of every 4 seconds when waiting for a switch press.

Note: If you have recorded a long sound using the TI program the pad would be lit all the duration of the sound. This could dramatically reduce battery life.


Recording with an MP3 player or external microphone

If you want to record a long piece of music or narration the onboard microphone is not ideal because of background noise such as doors slamming. The ideal solution is to use a direct connection using a wired lead.

Record with the wired lead



If you wish to record music or narration directly from your MP3 player connect one end of the supplied jack lead into the headphone socket of your player and connect to the Doo-zy using the socket marked with the microphone icon.

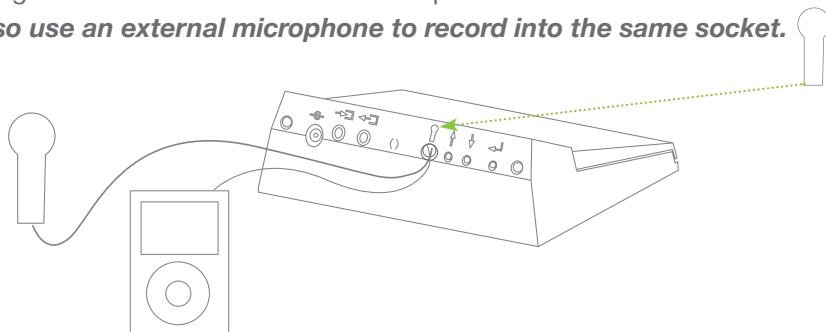We can also use an external microphone to record into the same socket.






Because the time involved to record music or narration we change the way Doo-zy records to make it easy.


Record music from an MP3 player

Plug your MP3 player into the switch. Now press the select button  until record  shows in the display. Now start the MP3 player by pressing play.

You will immediately hear the sound coming out of the Doo-zy speaker although the music will not be recorded. ***This is different to the way Doo-zy records with the built in microphone - you have 15 seconds to start recording before the record option is cancelled.***

The music will only be recorded when the switch pad is pressed  and the red  led is glowing. To stop the recording press the switch pad again .

Press the switch pad to playback your recorded sound .

Note: *The recorded sound may be very long and if using the timer output program it would be impossible to turn the sound off without a secret override. To immediately turn the sound off at any time during playback hold down the scroll up button  for more than 3 seconds.*








Infrared remote control learner




Use the Doo-zy to control infrared devices such as, toys, televisions, DVD players, virtually anything that uses infrared for control.



Doo-zy remembers a code from remote controls and then replays the code. Doo-zy can then be used as the remote control.

Learn an Infrared code

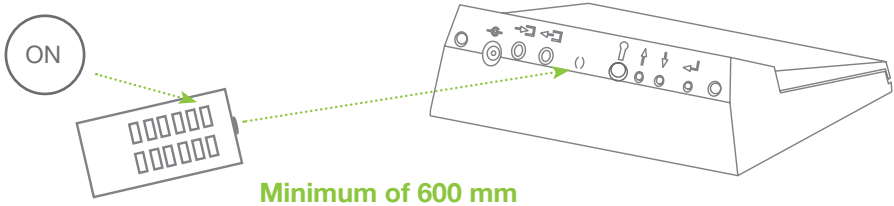Press the select button  until the display shows IR  Use the arrow up  to select learn  press the select button to confirm  and SET  is shown in the display. The IR  will flash in the display along with a flashing bar to indicate the switch is ready to learn the code.



Point the remote control towards the rear of the switch at a distance of around 600mm.

Press the button on the remote of the command you wish to copy. In our case the ON button for a TV. If the Doo-zy is receiving the code learn  will show in the display. When the code is copied the display will show SET  Now point the rear of the switch towards the device. Doo-zy will now control the ON function of the TV. You may wish to record a message that says TV when the switch is pressed.

If you do not wish to use the Infrared transmitter, set to number 1 in the infrared menu.

Please note it is important that the infrared beam is not broken or interrupted whilst being copied. The range of the beam will also depend on light conditions and obstacles within a room as infrared works with the line of sight. Both the infrared receiver and transmitters are in the rear of the switch.






Some code may not be able to be copied however we suggest a couple of tries before giving in.



Please note there is no such thing as a remote control that operates everything. However Doo-zy will control most infrared devices. The only way to find out is to try it!


Wireless Infrared transmitter function

* For use with the optional IR receiver

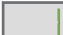Doo-zy has a built-in infrared transmitter which can send switch output information to the optional IR receiver. The switch should have a clear line of sight to the receiver. Sixteen infrared channels are available to use. Each receiver has to be paired with a switch. The factory default channel is zero, which will operate all 16 channels simultaneously. So, if you are using more than one it will have to be tuned to the receiver.

To choose which infrared channel to transmit press the select button  until the display shows IR  Use the arrow buttons   to select any one of the 16 channels to transmit  in this case one.

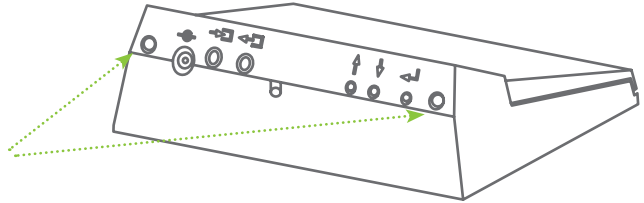

Press the select button  to confirm and SET  will be shown in the display.





IR Transmitters require a line of sight to receiver

Default and reset function

If at any time you lose track of the switch settings you can perform a factory reset.

To factory reset press the select button  until the display shows DEF 

Use the arrow buttons   to select RST  and the select button  to confirm. The switch is now factory reset.

Note: Full list of Factory defaults on page 29.

Switch output & programs

We need to understand how the switch output works.



We are only dealing with one of the two options at the moment and that is the 3.5 mm Jack socket output on the rear.

We shall start by explaining how the in-built programs affect the switch output.

The best way to do this is to have the switch nearby and connect a toy to the output jack socket and watch what happens!

This is where you connect items such as switch adapted toys using a 3.5mm jack lead just like Harry the Croc.

**'Harry the Croc'
switch adapted toy**

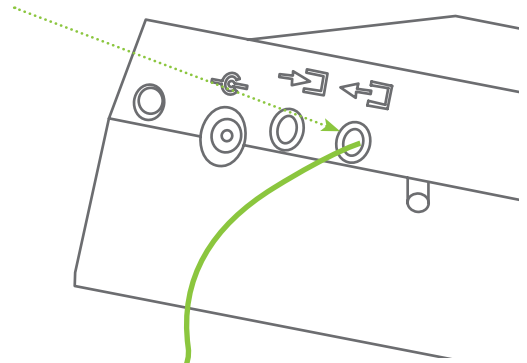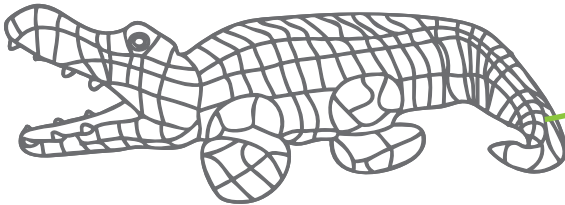

The switch output will react in different ways depending on which programs are set in the output menu indicated by **OUT** in the display. The following section explains the use of each program within the Output menu.

T1

Timed - Page 19

OPP

Opposite - Page 23

SEQ

Sequence - Page 21

DIR

Direct - Page 18

LATCH








Latch - Page 20

TOUT


Timed out- Page 24

Direct output

Step 1:- Record a sound. If you have forgotten how to go to page 10.





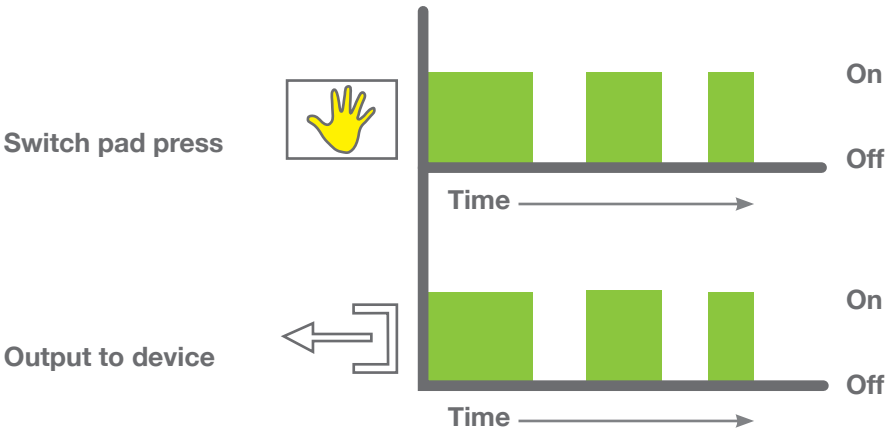


Step 2 :- Scroll to the  menu using the select button  and use the arrow buttons   to select DIR  press the select button  to accept and SET  is shown in the display. We have now selected direct output

When pressed the recorded sound will playback. If a toy is connected it will operate in synchronisation. As soon as you lift your hand off the pad both the toy and the sound will switch off immediately.



IN A NUTSHELL :- DIRECT ONLY OPERATES WHEN YOU LEAVE YOUR HAND ON THE PAD! YOU MAY KNOW THIS AS MOMENTARY.

Timed output


Step 1 :- Scroll to the  menu using the select button  and use the arrow buttons  to select TI  press the select button  to accept and SET  is shown in the display.

We have now selected the timed output.


Step 2 :- Record a sound. If you have forgotten how to go to page 10.

Now press the switch pad and switch will operate for the exact time length of the recording.



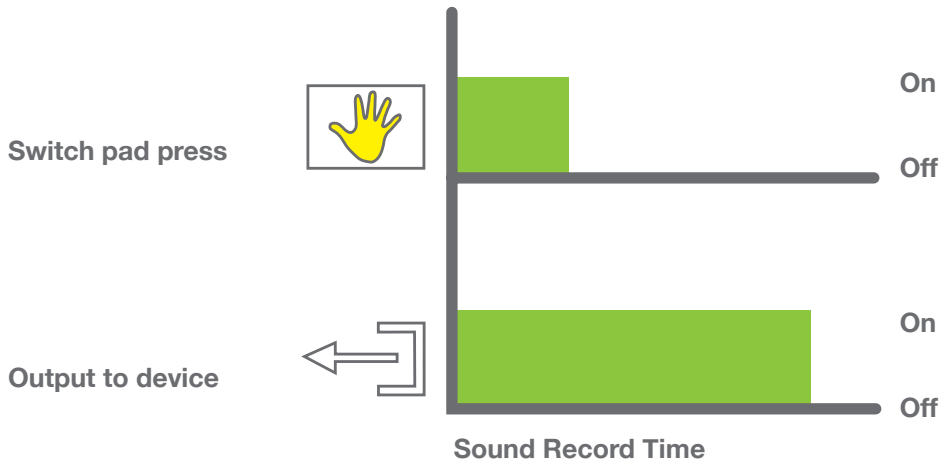

We have just compiled a **Timed program!**



IN A NUTSHELL :- THE SWITCH OUTPUT WILL OPERATE FOR THE DURATION OF THE RECORDED MESSAGE NO MATTER HOW LONG OR SHORT THE SWITCH PAD IS PRESSED.

The nature of the timed program makes it not possible to stop once the playback of a sound has started. This could be quite a problem if started by mistake! We therefore have a secret OFF switch. When playing back the sounds press and hold the arrow up  button for more than three seconds.


Latch output LAE

Step 1 :- Scroll to the OUT menu using the select button  and use the arrow buttons   to select LAT LAE . Press the select button  to accept and SET SET is shown in the display.

We have now selected the LAT or Latch output.

Step 2 :- Record a sound. If you have forgotten how to go to page 10.

Now press the switch pad and the switch will operate until the switch is pressed again, it has latched on.

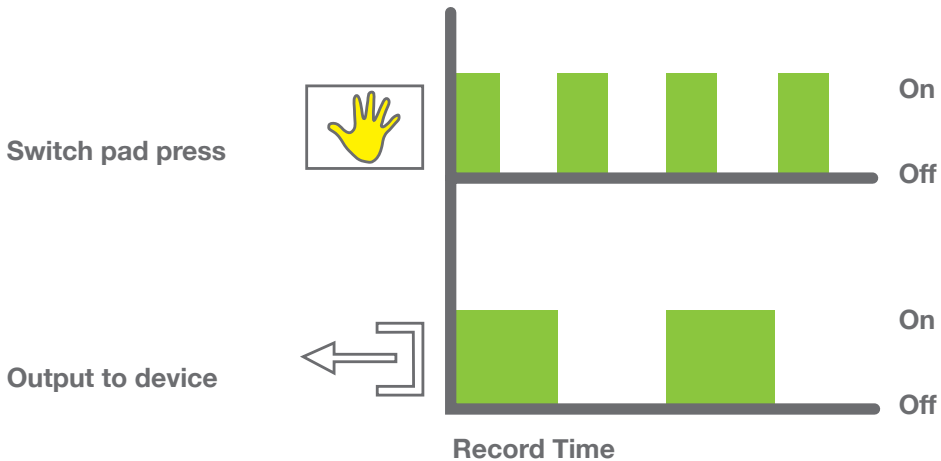








IN A NUTSHELL :- THE SWITCH OUTPUT WILL OPERATE INDEFINITELY UNTIL THE SECOND PRESS IS USED TO TURN IT OFF.

Sequence output

Sequence allows you to record a series of messages in sequence and plays back in the same sequence and on the final message activates the output. Record as many as you wish for as long as you wish.

The example here assumes that the on board microphone is being used to record.

Lets try this out!


Step 1 :- Scroll to the  menu using the select button  and use the arrow buttons   to select SEQ  press the select button  and SET  is shown in the display.

We have now selected the SEQ or Sequence output.


Step 2 :- Record a sequence.


Press the select button  until  shows in the display, now push down the switch pad and hold. Now talk clearly at about 20cm away from the mic  on the front panel, anything you say will be recorded. As long as you hold the switch pad down it will record.

Release the switch pad and press again for the second time and record the second message.


The display will show R2 

Release again and press for the third time with another message.

The display will show R3 

When you have finished press the select button once  to stop.

Now press the switch pad and it will play your recorded messages back in sequence every time the switch is pressed. On the final sequence (in our case R3) the output will switch on. In other words if a toy was connected when R3 activated the toy would switch on.

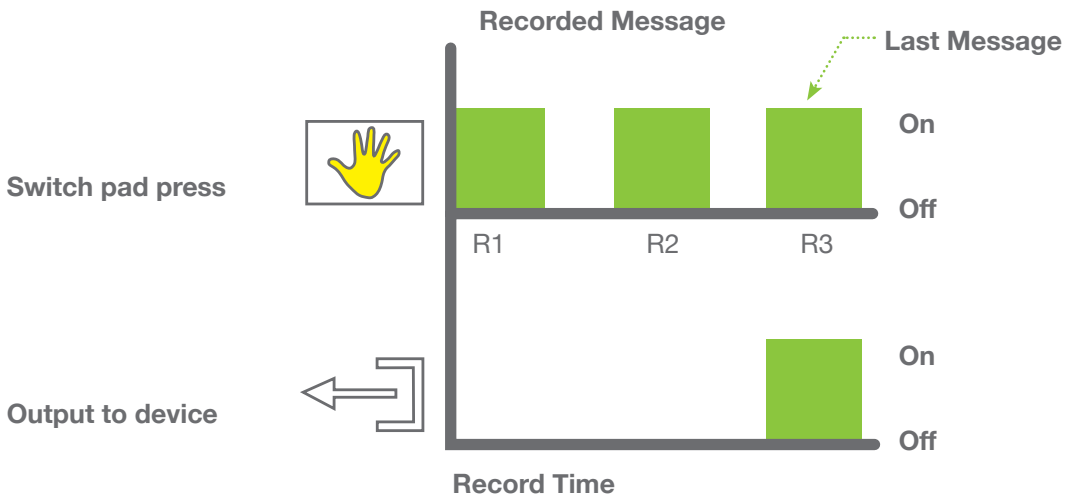





IN A NUTSHELL :- PLAYS THE SEQUENCE BACK IN ORDER RECORDED AND THE SWITCH OUTPUT WILL OPERATE ON THE LAST RECORDED MESSAGE R3.

Opposite output OPP


Strictly speaking the opposite does not control the output it merely changes how the switch pad interacts with the output. In fact the output is inverted and the switch pad operates the opposite way to normal with the output becoming active only when the switch is released. Lets try this out!

Step 1 :- Record a sound. If you have forgotten how to go to page 10.

Step 2 :- Scroll to the OUT menu using the select button  and use the arrow buttons   to select OPP OPP press the select button  to accept and SET SET is shown in the display. We have now selected the OPP or Opposite output.

Step 3 :- Now press the switch pad and hold down and you will notice the switch does not operate, now release. It now operates on release.

Note: When in opposite mode the switch will only work in timed.

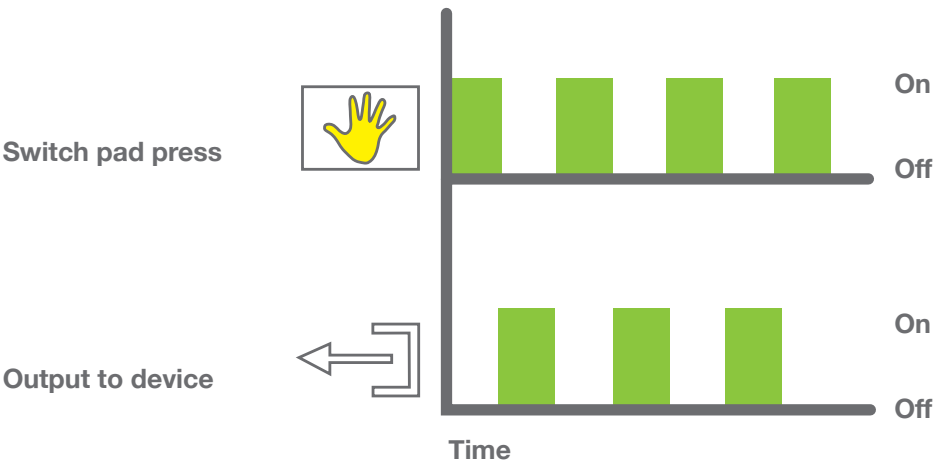





IN A NUTSHELL :- THE SWITCH OUTPUT WILL ONLY OPERATE WHEN THE SWITCH PAD IS RELEASED WHICH IS THE COMPLETE OPPOSITE TO WHEN PRESSED.

Timed out output


Timed out allows you to record a message and when the message is finished it immediately activates the output.

Step 1 :- Record a sound. If you have forgotten how to go to page 10.

Step 2 :- Scroll to the out menu using the select button  and use the arrow buttons   to select T0 . The Timer out is now selected.

Step 3 :- Now press the switch pad and the recording will playback. However the switch output will not operate until the exact moment the recorded sound has finished.

The switch output will stay on until the switch pad is pressed again or 30 seconds in time has elapsed.

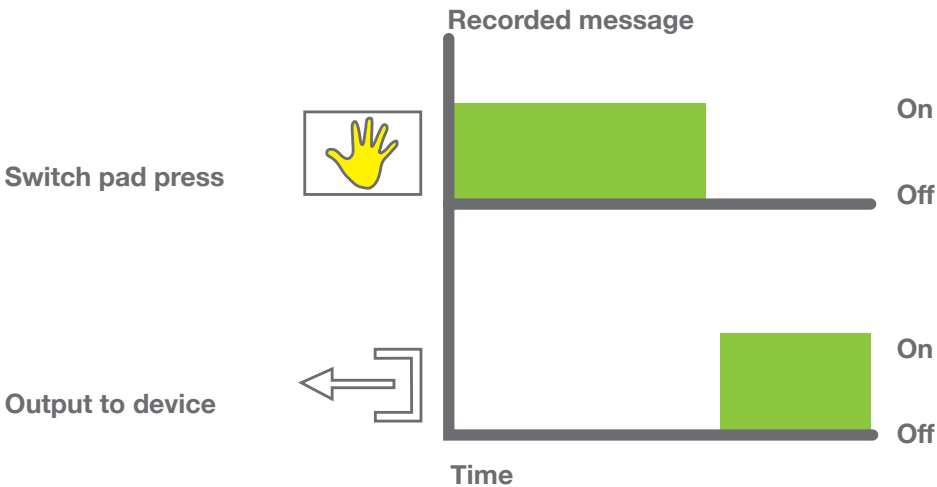

IN A NUTSHELL :- THE SWITCH OUTPUT OPERATES AS SOON AS THE RECORDED SOUND HAS FINISHED AND TURNS OFF WHEN THE SWITCH PAD IS PRESSED AGAIN OR A MINUTE HAS ELAPSED


Attachments & Mounting

Positioning your switch

In most cases the best position to use your switch is a flat surface. Doo-zy is fitted with non-slip rubber feet for maximum grip.


Fitting the low profile pad

For users with poor motor movement who may not be able to raise a hand sufficiently to operate the switch pad, a low profile pad clips to the front of the switch pad and lowers the pad to 10mm above a work surface and makes it easier to use.

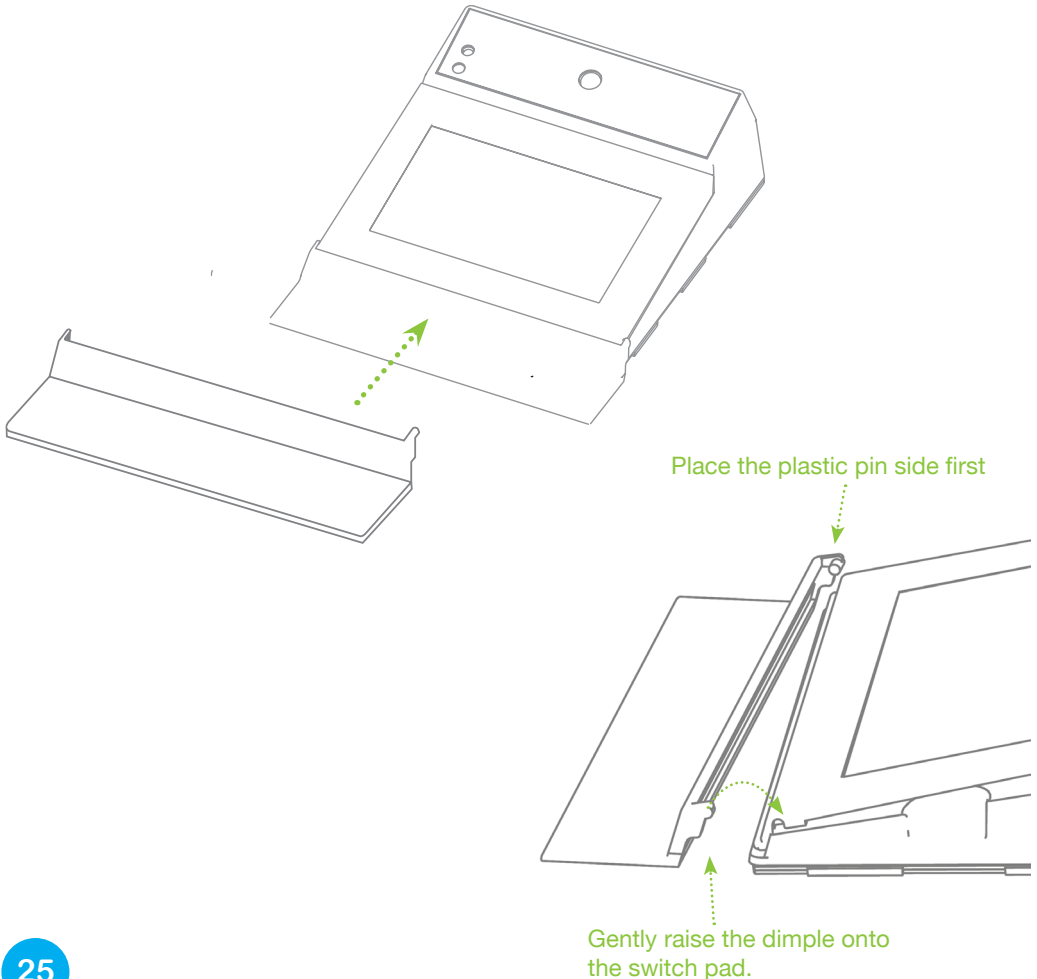


Graphic card slot

A graphic card slot is provided.
The maximum size paper that can be used is 125mm x 64mm. The switch pad LEDs illuminate the pad.

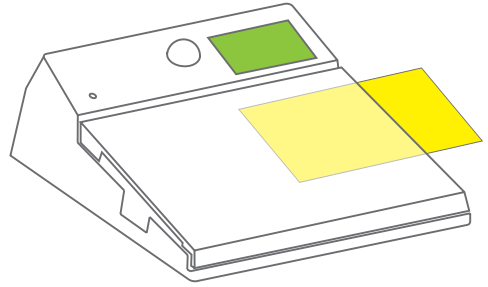


Cradle switch bracket

Magic Eye Beam

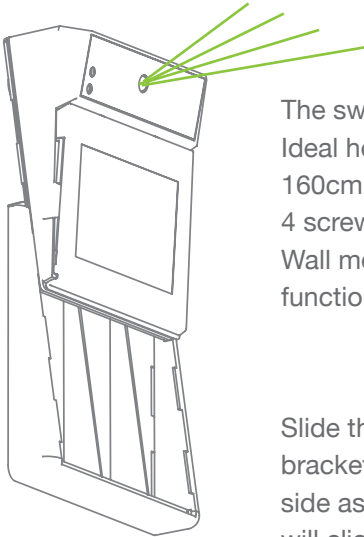

160 cm above floor level


The switch can be wall mounted. Ideal height above floor level 160cm. Easily wall fixed using the 4 screw holes shown. Wall mount is ideal for the Eye function.

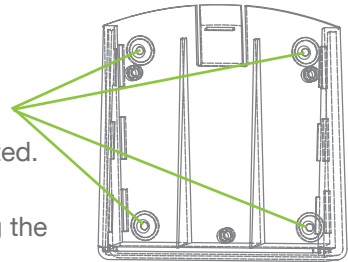

Slide the switch down the bracket using the grooves in the side as a guide. The cradle lock will click when in the correct position.

To remove from the cradle push the cradle lock clip whilst gently sliding the switch back up the cradle.

Press to release

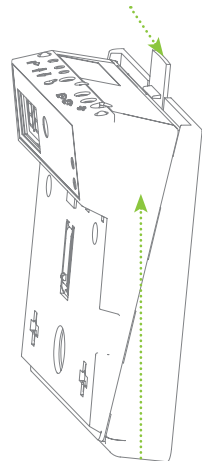


Slide upwards to remove from cradle

Cradle for table top use

On a table top the cradle is used to position the switch so the pad is horizontal. This makes it the perfect stable platform to place objects of reference so they simply don't roll off the edge!

Ideal for use with the opposite output or program, lift the object to activate.


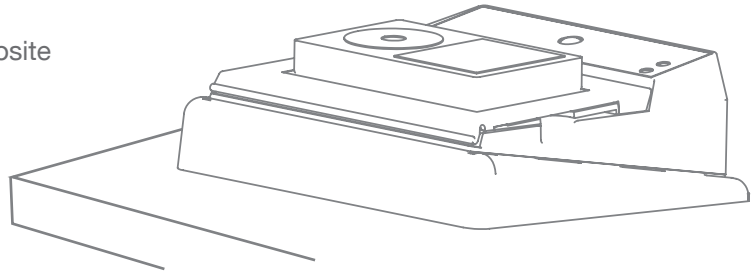


Mounting points for adjustable arm

Doo-zy has mounting points for the industry standard adjustable arm. The switch can be fixed in almost any position and whilst forming a stable base it can also be easily uncoupled to be used anywhere.

Image showing the triangular wall mount plate for use with the adjustable arm bracket. This fits to the rear of the cradle using 3 screws.

Use only 3 x M5 x 6 screws any others will damage the cradle.

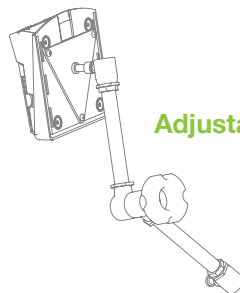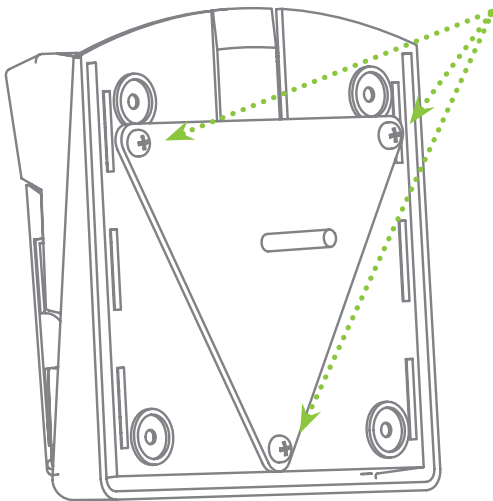

Adjustable arm bracket

Replacing the batteries & charging

Like all rechargeable batteries in time they will require replacement. When the batteries are exhausted the LED will usually show **BAT** even after a full 5 hour charge.

It is essential they are replaced with the correct specification observing the correct polarity.

The Doo-zy uses a special battery pack. Under no circumstances must any other battery pack be used otherwise permanent damage may occur.


Information of where to obtain a replacement is available at www.doo-zy.co.uk, The part number is 22091.

To replace the batteries

Flip the switch on its front. The battery compartment can now be viewed on the rear next to the speaker grill. Use a cross point screwdriver to unlock the battery compartment.

Gently ease the battery pack from the case allowing the battery connection cable to follow. Unplug the battery from the switch by gently pulling on the in line plug and socket. Dispose of the battery in a

responsible manner and connect the new battery being careful to ensure the plug and socket connector are pushed together fully. Refit the battery cover.

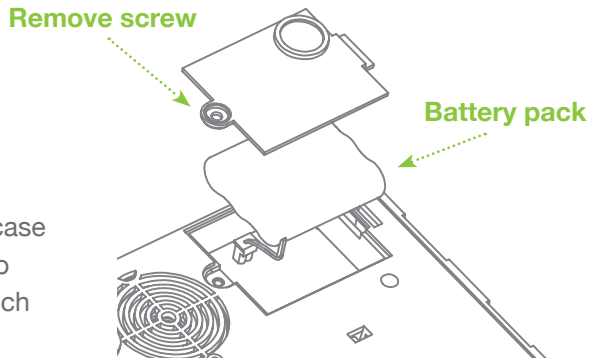

Doo-zy switch will charge fully over a 5 hour period, therefore we recommend this when the battery pack has been replaced.

Doo-zy switch uses a metal hydride rechargeable battery pack with a specification of 3.6 V DC @ 1800 ma



The batteries within Doo-zy should not be disposed of in household waste but taken to a suitable public disposal point.

Technical specification

Power Supply

110 – 240 vac – 12vdc @ 500 ma with a 2.1 mm dc plug centre pin hot.

Weight :- 80 grams

Doo-zy

Batteries :- Custom battery pack nickel metal hydride (NiMH) @ 3.6 Volt 1800 mA

Standby mode battery life :- 200 hours

Recharge time :- 6.5 hours

Expected battery recharge life :- 500 - 800 operations

Relay :- Via 3.5mm 2 pole jack socket. Normally open contact rated @ 1 amp

Memory :- micro SD

Recording time 6.5 hours with 1 GB SD card

Switch activation requirement force :- 4.5 Newtons

Switch pad activation weight :- 100 grams

Switch input :- 3.5mm 2 pole jack socket

Weight :- 427 grams

Size :- 150mm x 150mm x 40mm (H)

Factory Reset Defaults

Description	Displayed in menu	Default condition
Recorded sound	Rec	N/A
Volume	Vol	5
Output	Out	T1 (Timed)
Eye	Eye	Off
Vibration	Vib	Off
Display	Dis	Down
Illuminated Pad	Lit	Off
Infra Red	IR	1
Default	Def	N/A

Trouble shooting & hard reset

Questions and answers

Question - My switch will not turn on?

Answer - Ensure your batteries are charged fully by charging for at least 5 hours. If after 5 hours the switch will not work have your charger checked by a suitable qualified person. If the batteries are near the end of life this will be indicated with the BAT message in the LED display

Question - My LED display will not illuminate?

Answer - If none of the functions or the display appears not to be working chances are they have been disabled in the menu. Perform a full reset. Refer to page 16.

Question - My sound recordings aren't very loud even when set to volume 10?


Answer - When recording make sure you talk clearly no more than 20cm away from microphone on the front panel.

Question - When the user releases the switch pad the sound plays and not when pressed?

Answer - The output menu is set to opposite. Change this option in the OUT menu.

Question - I have strange and not recognised image in the LED display and the switch is acting very strange?

Answer - Doo-zy is a computer and can hang up or crash, this would be a good example. To correct this we would have to perform a hard reset or boot. On the underside of the Doo-zy we have a hard reset button. To perform a reset, push and hold for two seconds. Now turn the Doo-zy on in the usual way,

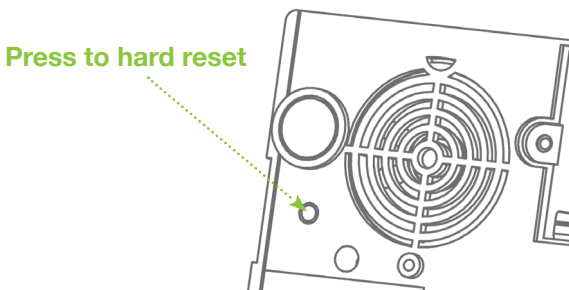

For a full trouble shooting guide go to www.doo-zy.co.uk

Doo-zy Compliance

Declaration European Union

Complies with the requirements of :-

EN 62115:2005 Electrical Toy safety

CE/EN 61000-6-3:2007 – EMC RoHS



Manufactured in China on behalf of :-

Jakmax ltd trading as Switch-ed

www.switch-ed.co.uk 00 44 (0) 845 299 65 66

enquiries@switch-ed.co.uk

Wilsden, Bradford, BD15 0LX.

United Kingdom.

Consumer Warranty

We strongly suggest you register your Doo-zy so we can keep you informed of any firmware issues and updates by going to our web site www.doo-zy.co.uk

This warranty covers Doo-zy, its hardware, accessories along with its power supply and its embedded software now together referred to as the product. Switch-Ed warrants the product will be free of defects in material and workmanship for a period of 12 months from the date of purchase from new.

To make a valid claim under this warranty you must notify Switch-Ed within 12 months of the date of purchase of the product and must return the product to us with 30 days of notifying us of the defect. After inspection if we accept the product is defective Switch-Ed (in its sole discretion) will either replace or repair the part causing the defect or replace the product without charge.

Exclusions

1. Defects in the product caused by accidental damage, unreasonable use.
2. Damage caused incorrect power supply and batteries as recommended by Switch-Ed but not limited to damaged or leaking batteries.
3. If the product is resold.

How to claim

Contact Switch-Ed by checking the latest contact details at www.switch-ed.co.uk
The mailing address will be in the contact us section of the website. You may want to ring us to discuss your defect before sending anything.

- 1. Package the product.** If you have thrown the original packaging away send the product in a sturdy box that will protect the product, you accept that Switch-Ed will not be held responsible for the product in transit.
- 2. Provide a description.** Let us know what is the defect.
- 3. Attach a proof of purchase.** A copy is fine.
- 4. Ensure the product arrives within 30 days of informing us of the defect.**