



1. Product Name

Oral Motor Kit

2. Product Code

21326

3. Colour

Multicoloured components

4. Brief Description

A kit of different products that can be used for a variety of oral motor focused interventions.

5. Contents

Typically includes (although items in the kit may vary):

- 1 X Drinking Tumbler (turtle tumbler with straw)
- 1 X Drinking Bottle (clown fish drinking bottle)
- 1 X Zoo Bubbles
- 1 X flavoured Tongue depressors
- 1 X Silly Straw
- 1 X Puppy Slide Flute

1 X Water Flutes
1 X Water Warbler
1 X Ball Blower
1 X Harmonica
1 X Inflatable Tongue
1 X Metal Kazoo
1 X Train Whistle
1 X Recorder
1 X Blow Light
1 X Hooting Owl
1 X Glow in the Dark Straws
1 X DIY Straws
1 X Hornbill Water Whistle
1 X Chew Lolli Knobbly Blue
1 X Chewy Orange
1 X Chewy (Tough)
1 X Chewy A's and Z's

6. Snoezelen® Stimulations

- Visual
- Tactile
- Taste
- Aroma

7. Best Use

Use under supervision, creating purposeful activities concentrating on clients' needs, e.g. strengthening oral motor muscles, oral awareness, providing oral motor input, blowing, sucking, chewing etc.

8. Compatible ROMPA® Products

There are many other items that may be useful to use with your client as purposeful oral motor activities. These have not been included because they are readily available on the high street. Whoopie cushions, rocket balloons, popping candy, sour sweets, feathers, ice cube trays, windmills, balloons, party blowers, ping pong balls, flavoured lip gloss etc....our Pinterest page has a variety of oral motor ideas for activities.

<https://www.pinterest.com/rompaltd/>

Many multisensory items can be used passively or interactively. If you wish to increase sensory stimulation for your client/s, please refer to our catalogue or website www.rompa.com for a great selection of items that provide a huge variety of physical, cognitive, olfactory, tactile, audible and visual input.

9. Starting Up

- Unpack
- Engage with client

10. Detailed Description

This kit offers a variety of items that can be used for many different oral motor activities.

11. Safety

ALWAYS SUPERVISE THE USE OF THESE PRODUCTS

- Kits contain small parts and therefore use must be supervised at all times.
- Take time to familiarise yourself with the instructions provided for each individual item.
- None of these products are intended for use as toys. They have been provided for therapeutic use and interventions around oral motor work and by their nature they carry the risk of choking. Please ensure that each product is used following risk assessment in a controlled manner and under close supervision.
- All items should be carefully and regularly checked for wear and tear. Discontinue use if any item is damaged and is assessed to present risk to the user.
- Remove all packaging and either store this in a safe place out of reach of clients or dispose of this responsibly. Plastic bags and wrapping can be hazardous to children and vulnerable people.
- Make sure all adjustments and parts are secure and tight before use.
- Keep all items away from sources of heat and fire.

12. Technical Specification

- Should you require additional information for a specific component, please contact us.

13. Installation

- Unpack
- Decide which activity will be beneficial to your client.
- Discard packaging, retaining instructions for use.
- Decide where and how to carry out oral motor activities.
- Carry out intervention under close supervision

14. Care and Maintenance

- Organise a regular cleaning and maintenance schedule.
- Should any wear be found, discontinue use and replace any worn parts immediately.
- Store in a safe place out of direct sunlight when not in use.
- Store out of reach of clients – do not allow unsupervised use.
- Items can be cleaned carefully with a mild soap solution and dried quickly and effectively (please, however refer to packaging instructions for individual items and do not allow wooden items to soak).
- Do not use harsh, abrasive solvent or bleach cleaning products.
- Ensure items are rinsed and dried thoroughly before reusing, especially those that are put in the mouth. We have intentionally included a lot of items in this kit to allow items to be used by one individual for hygiene purposes.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at producthelp@rompa.com