



## 1. Product Name

Squeezer

## 2. Product Code

21544

## 3. Colour

Likely to be as shown

## 4. Brief Description

Provides deep pressure for additional sensory input.

## 5. Contents

- 1 x Squeezer (supplied already assembled)

## 6. Snoezelen® Stimulations

- Pressure

## 7. Best Use

For use by a suitably qualified professional, in a safe place on a flat, stable surface. For indoor use only.



## **8. Compatible ROMPA® Products**

ROMPA® offer a range of Sensory Integration suspension products such as safety mats, swings, gliders, nets, bolsters etc. Please visit [www.rompa.com](http://www.rompa.com) for details about our full range.

## **9. Starting Up**

There are no short-cuts to setting up Sensory Integration products. Read all instructions thoroughly before using the Squeezer and keep the instructions in a safe place.

## **10. Detailed Description**

Provides deep pressure for additional sensory input. Masterfully crafted with a Baltic birch wood frame. Tension and pressure are easily adjusted to each user's specific needs. Cushioned foam rollers are covered in an easy-care polyurethane vinyl and they squeeze and release pressure evenly and consistently. With hand holds for portability. Colours may vary. Weight limit: 45kg (100 lbs)

## **11. Safety**

### **Always supervise the use of this product.**

This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.

- Do not exceed the 45kg (100 lbs) weight limit.
- This equipment is intended for professional clinical use only under supervised conditions by a trained specialist. Use without proper supervision or knowledge is potentially dangerous and should not be undertaken or permitted. Before using these products, make sure you know your limitations and that of your equipment.
- Client safety is dependent on the appropriate choice of activity to match, yet challenge physical ability.
- The layout of the activity area, proper installation and maintenance of equipment is critical – use this product appropriately and safely.
- Do not use or store outside.
- Take time to familiarize yourself with the use and maintenance of this equipment before using and file these instructions in an accessible location. These instructions must be read and be familiar to anyone who is using this equipment.
- Check all points where movement occurs for wear of any kind. All wear points should be checked regularly and recorded on the maintenance checklist enclosed to monitor their condition. Worn parts should be replaced or repaired immediately.
- Use a safety helmet and wall/floor mats of an appropriate thickness as required.

- Risk assess before use. Inspect equipment for wear and tear before use and discontinue use if any doubt.
- Allow plenty of room around the Squeezer to allow clients to safely enter and exit the rollers.


## 12. Technical Specification

Size: 85L x 60W x 38cmH

Weight limit: 45kg (100 lbs)

## 13. Installation

- Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact ROMPA® immediately.
1. The Squeezer is supplied already assembled, although adjustments may be necessary to reduce or increase the compression tension. To adjust this, locate the tab hooks and the red rubber tube straps on each side of the Squeezer. There are two rubber tube straps on each side. Compression levels can be adjusted by how shallow or deep the tube strap is connected to the tab hooks.

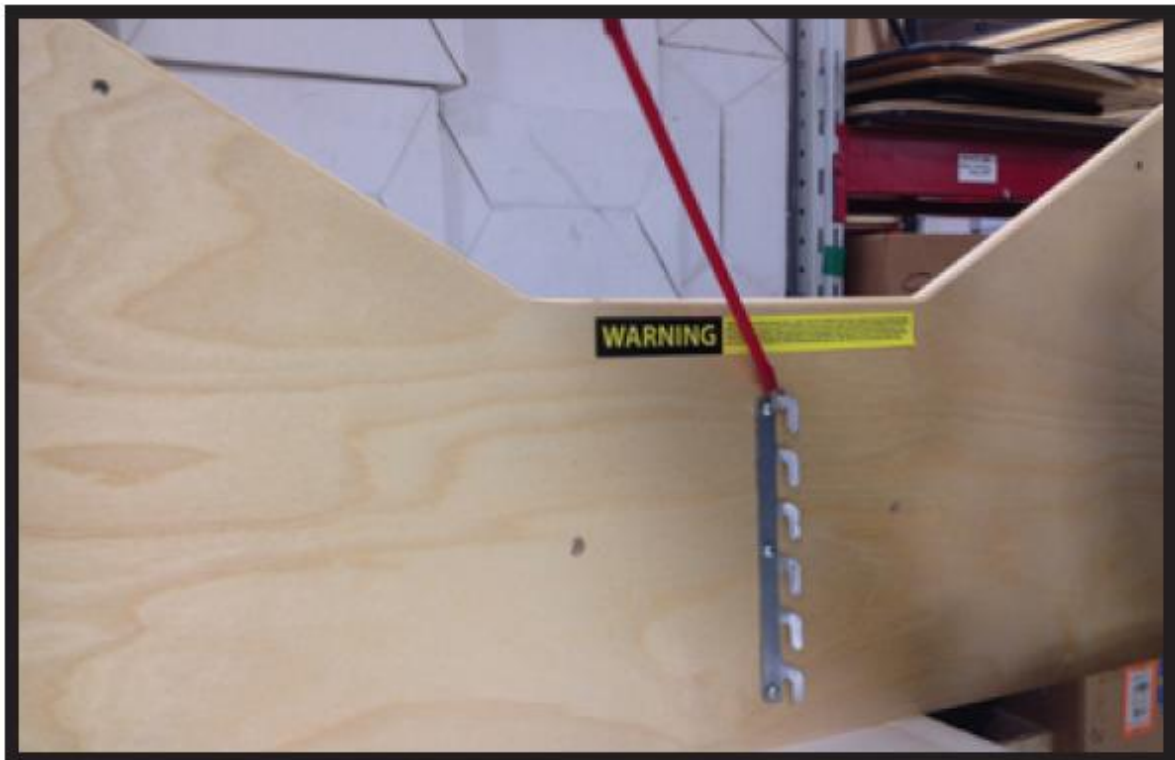

2. Attach the tube straps to the lower most tab hook for the strongest compression. Connect the tube straps to the higher tab hooks for lesser compression.



3. Make sure that all adjustments are secure and tight before use.
4. Check all materials, stitching and connections for wear before using.

## **14. Care and Maintenance**

1. Check for client allergies before using any chemical cleaners.
2. Wipe down with a soft cloth dampened with a mild soap solution and allow to air-dry naturally. Do not use large amounts of liquid and do not immerse any part of this product in water.
3. Use and store in a dry environment.

## **15. Troubleshooting**

*Not applicable*

Further copies of these instructions can be downloaded at [www.rompa.com](http://www.rompa.com)

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at [producthelp@rompa.com](mailto:producthelp@rompa.com)

