



## **1. Product Name**

Vertical Bouncer Kit

## **2. Product Code**

21567

## **3. Colour**

As shown

## **4. Brief Description**

Use with any swing for added bounce! Specifically designed to enable controlled bouncing and vestibular stimulation with any therapy equipment you attach to it.

## **5. Contents**

- 1 x Vertical Bouncer Kit:
  - 2 x Single Vertical Bouncer units
  - 2 x Double Vertical Bouncer units
  - 2 x locking Safety Snaps



## **6. Snoezelen® Stimulations**

- Vestibular

## **7. Best Use**

Use from a safe and appropriate suspension system.

## **8. Compatible ROMPA® Products**

13672 – Hanging Bracket

21559 – Safety Rotational Device

21570 – Safety Snap

See [www.rompa.com](http://www.rompa.com) for compatible swings, helmets, mats and a large range of other Sensory Integration products

## **9. Starting Up**

- There are no short-cuts when setting up and using Sensory Integration equipment. Please keep these instructions in a safe place and refer to them before each use.

## **10. Detailed Description**

Use with any swing for added bounce!

Specifically designed to enable controlled bouncing and vestibular stimulation with any therapy equipment you attach to it. Includes 2 Single Vertical Bouncers, two 2 Double Vertical Bouncers, and 2 Safety Snaps. Reinforced elastic loops are sewn onto the ends of the Single and Double Vertical Bouncers. Weight capacity: 129kg (285 lbs)

## 11. Safety

- Always supervise the use of this product
- Always use a Rotational Device to help prevent wear of your swing and the suspension points.
- Do not exceed the weight capacity: 129kg (285 lb).
- Sensory Integration products are intended to be used by professionals.
- Use safety mats on the floor, as well as on the walls as necessary.
- Use a safety helmet, and any other appropriate safety measures for your client's needs.
- Use only steel safety snaps as aluminium safety snaps wear more quickly.
- For indoor use only.
- Check all hardware and stitching before each use to check these are intact.

This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.

## 12. Technical Specification

Weight capacity: 129kg (285 lb)

Reinforced elastic loops are sewn onto the ends of the Single and Double Vertical Bouncers

## 13. Installation

Allow plenty of unobstructed space around your swing. Suspended equipment needs a minimum ceiling height of 2.5m (8 feet). Suspended equipment should be positioned a clear distance of 1.8m (6 feet) from walls and obstacles.

### **Safety Snaps**

Our Safety Snaps can be locked.

1. Turn the locking nut to open the snap (to slide this nut down to allow the snap to be opened). Turn the nut in the opposite direction to slide it up and over the opening to lock it.




### Vertical Bouncer Kit

1. Open one Safety Snap on the Vertical Bouncer Kit and unhook it to flatten out the bouncer.



2. Select the desired single or double bouncer attachments and disconnect the unwanted accessories from the Safety Snap.

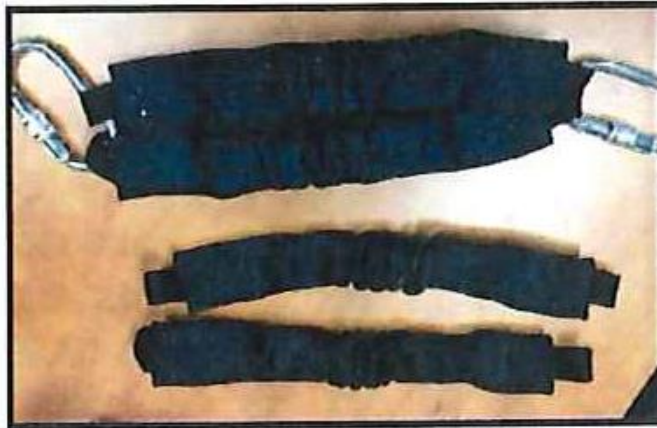

3. Attach one of the Vertical Bouncer Kit Safety Snaps to the reinforced loop on the top of the swing.



4. Connect the Safety Snap on the other end of the Vertical Bouncer Kit to your secured end point of suspension.



## **14. Care and Maintenance**

1. All the parts of your suspended products should be checked on a regular basis (at least monthly, and more if the swing is used often). Use a maintenance sheet (such as the example one provided) to record these checks.
2. Check all the parts for signs of wear and tear before each use and replace these as appropriate. For example, check the fabric parts for loose stitching.
3. Use and store this product indoors, in a safe place.
4. Dust or wipe the components clean with a damp cloth. Do not immerse in water. Allow to dry before use.

## **15. Troubleshooting**

*Not applicable*

Further copies of these instructions can be downloaded at [www.rompa.com](http://www.rompa.com)

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at [producthelp@rompa.com](mailto:producthelp@rompa.com)

