



1. Product Nam

TheraGym[®] Sling Swing

2. Product Code

21580

3. Colour

As shown

4. Brief Description

The Sling Swing provides stimulation in a prone or seated position with its versatile aerial hammock suspension and quality construction. Safe for everyone from children to adults. Includes one safety snap. Weight capacity: 136kg (300 lbs).

5. Contents

1 x Sling Swing 2 x locking Safety Snaps

6. Snoezelen® Stimulations

• Vestibular

7. <u>Best Use</u>

Use from a safe and appropriate suspension system.

8. Compatible ROMPA® Products

21928 – Hanging Bracket 21559 – Safety Rotational Device 23011 (21570, 21590) – Safety Snap 21567 – Vertical Bouncer Kit

See <u>www.rompa.com</u> for mats and a large range of Sensory Integration products

9. Starting Up

• There are no short-cuts when setting up and using Sensory Integration equipment. Please keep these instructions in a safe place and refer to them before each use.

10. Detailed Description

Provides stimulation in a prone or seated position with its versatile aerial hammock suspension and quality construction. Safe for children and adults. Includes one locking Safety Snap. This product can be suspended from one suspension point or from two. Weight capacity: 136kg (300 lbs).

11. Safety

- Always supervise the use of this product
- Always use the Rotational Device to help prevent wear of your swing and the suspension points.
- Do not exceed the working load of 136kg (300 lb).
- Sensory Integration products are intended to be used by professionals.
- Use safety mats on the floor, as well as on the walls as necessary.
- Use a safety helmet, and any other appropriate safety measures if required for your client's needs.
- Use only steel safety snaps as aluminium safety snaps wear more quickly.
- For indoor use only.
- Check all hardware and stitching before each use to check these are intact.

This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.

12. <u>Technical Specification</u>

 Working load:
 136kg (300 lb)

 Size:
 124cm L x 76cmW (49"L x 30"W)

13. Installation

Allow plenty of unobstructed space around your swing. Suspended equipment needs a minimum ceiling height of 2.5m (8 feet). Suspended equipment should be positioned a clear distance of 1.8m (6 feet) from walls and obstacles.

Safety Snaps

Our Safety Snaps and the included Carabiner can be locked.

1. Turn the locking nut to open the snap (to slide this nut down to allow the snap to be opened). Turn the nut in the opposite direction to slide it up and over the opening to lock it.



Sling Swing

- 1. Slide the two Suspension Straps (located at the top of the swing) onto the Safety Snap.
- 2. Connect the Safety Snap to your point of suspension and lock the Safety Snap closed.





14. Care and Maintenance

- 1. All the parts of your suspended products should be checked on a regular basis (at least monthly, and more if the swing is used often). Use a maintenance sheet (such as the example one provided) to record these checks.
- 2. Check all the parts for signs of wear and tear before each use and replace these as appropriate. For example, check the Sling Swing for loose stitching.
- 3. Use and store this product indoors, in a safe place.
- 4. Dust or wipe the components clean with a damp cloth. Do not immerse in water. Allow to dry before use.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at producthelp@rompa.com

THERAGYM[®] SLING SWING 21580, January 2020 Copyright ROMPA[®] Ltd



SENSORY INTEGRATION MAINTENANCE CHECK LIST

PRODUCT:

ТО СНЕСК	Date / S (Satisfactory) or R (Repair/replace)													