



1. Product Name

Scooter Ramp

2. Product Code

21582

3. Colour

Natural wood finish

4. Brief Description

Ideal for promoting postural extension, motor co-ordination and fun activities.

5. Contents

- 1 x hinged platform
- 2 x bumper rails
- 1 x ramp slide
- 1 x ramp stabiliser



The product is shown above with a scooter board and rope pulley system (neither included)

6. Snoezelen® Stimulations

- Balance

7. Best Use

For use by a suitably qualified professional, in a safely-located and appropriate place.
For indoor use only.

8. Compatible ROMPA® Products

Please visit www.rompa.com for details about our full range, including wall and floor cushions.

- 14589 Basic Roller Board
- 20050 Round Roller
- 18363 Square Scooter
- 17812 Connect-a-Scooter
- 13734 Adjustable Scooter Board with Tow Strap by ROMPA®
- 16018 Puzzle Block (to knock down)
- 18354 Beginners Bowling Set (to knock down)

9. Starting Up

There are no short-cuts to setting up Sensory Integration products. Read all instructions thoroughly before installing and using the Scooter Ramp and keep the instructions in a safe place.

10. Detailed Description

Ideal for promoting postural extension, motor co-ordination and fun activities. The wide top platform allows users to turn smaller scooters around. Made from extremely strong Baltic birch. Folds flat for easy storage. No tools are required for assembly. Weight capacity: 113kg (250 lbs).

11. Safety

Always supervise the use of this product.

This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.


- Do not exceed the 113kg (250lbs) weight capacity.
- This equipment is intended for professional clinical use only under supervised conditions by a trained specialist. Use without proper supervision or knowledge is potentially dangerous and should not be undertaken or permitted. Before using these products, make sure you know your limitations and that of your equipment.
- Client safety is dependent on the appropriate choice of activity to match, yet challenge physical ability.
- The layout of the activity area, proper installation and maintenance of equipment is critical – use this product appropriately and safely.
- Do not use or store outside.
- Take time to familiarize yourself with the use and maintenance of this equipment before using and file these instructions in an accessible location. These instructions must be read and be familiar to anyone who is using this equipment.
- All parts of the product should be thoroughly checked at least on a monthly basis and before use.
- Check all points where movement occurs for wear of any kind. All wear points should be checked regularly and recorded on the maintenance checklist enclosed to monitor their condition. Worn parts should be replaced or repaired immediately.
- Clients should wear a safety helmet as necessary.
- Floor mats of an appropriate thickness should be positioned properly around all equipment. Wall matting/cushioning should also be considered if appropriate.
- Risk assess before use. Inspect equipment for wear and tear before use and discontinue use if any doubt.

12. Technical Specification

Size: ramp 145cmL; platform 76 x 76 x 35.5cmH
Weight capacity: 113kg (250lbs)
Materials: Baltic birch wood

13. Installation

- Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact ROMPA® immediately.

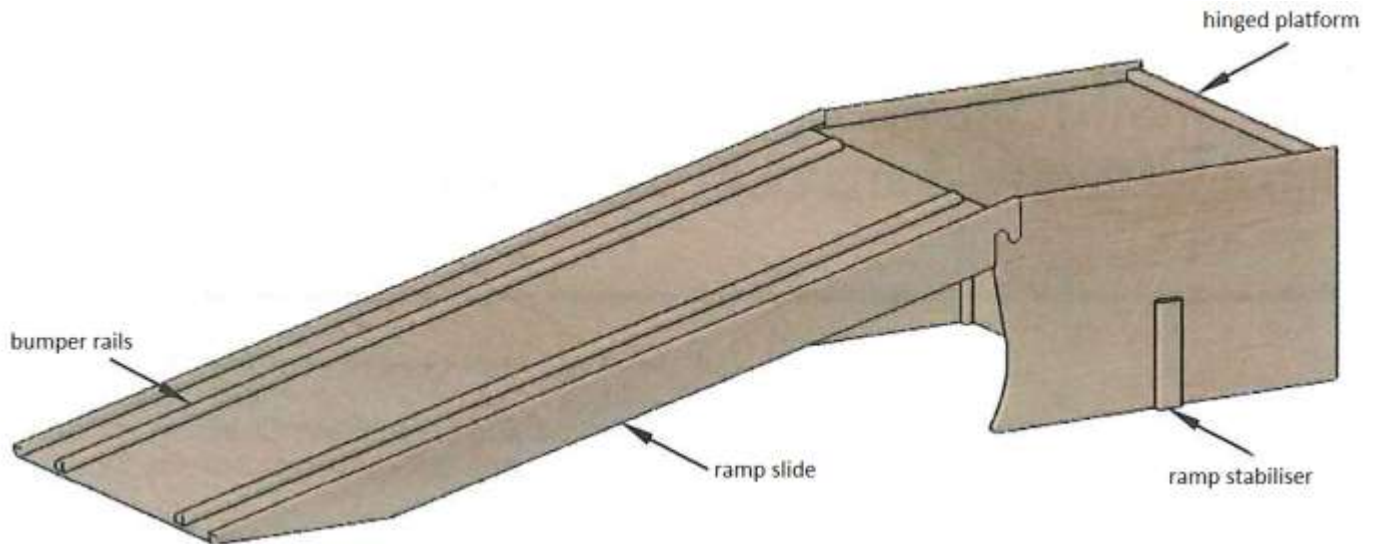

1. Unfold the hinged platform.
2. Slide the stabilizer into both slots on the hinged platform.
3. Attach the ramp slide.
4. Use the bumper rails on the ramp slide if desired. The holes for these are pre-drilled. Use the shelf pins included to attach the rails (push these into the pre-drilled holes). Ensure that the ramp bumper rails are assembled correctly. One side of the ramp and its parts are marked with a 1 and the other side with a 2. The easy way to be sure everything lines up is to match the numbers. If a rail does not match up, then it should just be turned around and if they still do not line up, it simply goes on the other side.

14. Care and Maintenance

1. Wipe down with a soft cloth dampened with a very mild soap solution and allow to air-dry naturally. Do not use large amounts of liquid and do not immerse any part of this product in water. If this product were exposed to wet conditions or left outside, the moisture would cause the wood to swell and break apart.
2. Use and store in a dry environment.

15. Troubleshooting

If the rails are not fitting correctly into the pre-drilled holes and the rail seems to be lifting, looking similar to the photographs below it simply means that the ramp has been incorrectly assembled.



One side of the ramp and its parts are marked with a 1 and the other side with a 2. The easy way to be sure everything lines up is to match the numbers. If a rail does not match up, then it should just be turned around and if the shelf pins and pre-drilled holes still do not line up, it simply goes on the other side.

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at producthelp@rompa.com

