



## **1. Product Name**

Deep Pressure Vest

## **2. Product Code**

21600 Extra Extra Small

21601 Extra Small

21552 Small

21545 Medium

21556 Large

21564 Extra Large

## **3. Colour**

Colours vary

## **4. Brief Description**

Provides proprioceptive feedback and a sense of stability. Made of breathable, latex-free perforated neoprene that's designed to provide deep pressure to the body and to help the user maintain a comfortable body temperature.

## **5. Contents**

1 x Deep Pressure Vest

## 6. Snoezelen® Stimulations

- Tactile
- Proprioception

## 7. Best Use

Provides proprioceptive feedback and a sense of stability. Made of breathable, latex-free perforated neoprene that's designed to provide deep pressure to the body and to help the user maintain a comfortable body temperature. Features 2 side and 2 shoulder closures. Colours may vary. Wash before wearing.

## 8. Compatible ROMPA® Products

If you wish to increase sensory stimulation for your client/s, please refer to our catalogue or website [www.rompa.com](http://www.rompa.com) for a great selection of items that provide a huge variety of physical, cognitive, olfactory, tactile and visual input.

## 9. Starting Up

- Choose the correct size vest according to client's dimensions.
- Take out of the bag and undo the Velcro® fastenings.
- Secure around your client (e.g. pass over the head) and adjust where necessary to ensure a snug but comfortable fit.
- Keep the bag in a safe place, out of reach, for storage after use.

## 10. Detailed Description

Provides proprioceptive feedback and a sense of stability. Helps improve body awareness. Made of breathable, latex-free perforated neoprene that's designed to provide even deep pressure to the body and to help the user maintain a comfortable body temperature. Features 2 side and 2 shoulder closures. Colours may vary. Wash before wearing.

## 11. Safety

- Always supervise the use of this product
- This product is not a toy and should never be treated as a toy.
- Keep away from excessive heat, including flames, heaters, matches, cigarettes etc.

## 12. Technical Specification

Perforated Neoprene, latex-free, Velcro® fastenings

Size:	XXS	30.5L x 51cm W
	XS	35.5L x 61cmW
	S	43L x 76cmW
	M	51L x 91cmW
	L	58L x 106.5cmW
	XL	66L x 122cmW

## 13. Installation

- Remove item from packaging
- Fit snugly but comfortably
- The Deep Pressure Vest has two side and shoulder closures.
- Open the two side closures and secure the two shoulder closures before placing the top of the vest over the head of a client. If the opening between the shoulder areas of the vest cannot be placed over the head of the client, you can open one side of the shoulder closure and put the vest on from the side.
- Fasten all the closures and adjust the amount of overlap on the side and shoulder closures to adjust the amount of pressure the client will experience. When adjusted properly, you should be able to pinch the fabric and pull it slightly away from the client's body. If this proves to be too difficult, it may be fastened too tightly. If you can pull the fabric away from the client's body too easily, you may need to adjust the top a little tighter.

## 14. Care and Maintenance

- Before washing, secure Velcro®.
- Wash with cold water & air dry.
- Separate the closures with care – do not separate these open quickly and forcibly as this is likely to damage the Vest.

## 15. Troubleshooting

*Not applicable*

Further copies of these instructions can be downloaded at [www.rompa.com](http://www.rompa.com)

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at [producthelp@rompa.com](mailto:producthelp@rompa.com)