



**Haley's Joy®**  
Swing Therapy Anywhere!

## Reagan's Ride Instruction Manual

### **WARNING**

**READ ENTIRE MANUAL BEFORE USE. THIS SWING IS NOT A TOY.**

**THIS SWING IS ONLY TO BE USED UNDER ADULT SUPERVISION. CONSULT WITH A CHILD'S THERAPIST ON HOW TO USE.**

**THIS SWING IS DESIGNED AS A THERAPEUTIC TOOL FOR A CHILD WITH DEVELOPMENTAL NEEDS. USE OF THE SWING SHOULD ALWAYS BE IN A CONTROLLED FASHION WITH AN ADULT'S HAND ON THE SWING, GUIDING THE MOTION.**

**THE SWING SHOULD ALWAYS BE GUIDED BY PLACING THE HANDS ON THE SWING, ITSELF. PUSHING ON THE CHAINS COULD RESULT IN EARLY FAILURE OF THE CHAINS AND VINYL.**

**ONLY UTILIZE THE SWING WITH ONE INDIVIDUAL AT A TIME, AND ALL HANDS AND FEET SHOULD BE KEPT OUT FROM UNDER THE SEAT.**

**THIS SWING IS A LOW MOTION DEVICE; KEEP THE SWING'S OCCUPANT / SEAT WITHIN THE PERIMETER OF THE FOUR LEGS. FAILURE TO DO SO CAN RESULT IN TIPPING OF THE SWING.**

**NEVER ALLOW THE INDIVIDUAL TO STAND ON / IN THE SWING; EVEN DURING THERAPY.**

**THE SWING IS DESIGNED TO BE USED CLOSE TO THE GROUND.**

**THERE IS NO SEAT BELT. CARE MUST BE TAKEN NOT TO ALLOW THE CHILD TO FALL OFF THE SWING.**

**CHOKING HAZARD: DO NOT ALLOW EATING OR DRINKING ON THE SWING!**

**STRANGULATION HAZARD: DO NOT ATTACH STRINGS OR ROPES TO THE SWING OR PLACE THE SWING NEXT TO ANY STRINGS OR ROPES.**

**UNSUPERVISED USE OF THIS SWING CAN RESULT IN SEVERE INJURY. WHEN NOT IN USE, THE SWING MUST BE DISASSEMBLED AND STORED IN A SAFE PLACE.**

**WORKING LOAD:** 150 lbs for the Reagan's Ride 1, 250 lbs for the Reagan's Ride 2, and 300 lbs for the Reagan's Ride 3.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please keep this Instruction Sheet in an accessible location. The Instruction Sheet must be read and familiar to anyone who is using the Reagan's Ride. If this Instruction Sheet is misplaced, please visit the product page on the FlagHouse website or email ([haleysjoy@flaghouse.com](mailto:haleysjoy@flaghouse.com)) for a replacement.

## IMPORTANT

Please check equipment immediately for any damage that may have occurred in shipping. Contact FlagHouse immediately if there is a damaged piece 800.793.7900 (US & International) or 800.265.6900 (Canada).

## GENERAL INFORMATION

The Reagan's Ride Swings are designed to give the users another option to the platform seat. This upright seat is totally enclosed on both sides and the back, giving the child a feeling of security and closeness. The open front allows for easy seating / removal of the child and access for therapy with the child.

## SWING CONSTRUCTION DETAILS

- The Reagan's Ride Swings are made of a durable soft micro suede material on the inside layer and a tough durable woven acrylic nylon material on the outside layer.
- All load-carrying seams are double stitched for strength.
- Features a unique lightweight horseshoe-shaped support structure that gives full swing support on all sides. There is also padding surrounding this structure for extra protection.
- There is a 4" thick pillow on Reagan's Ride 1 while Reagan's Ride 2/3 has a 6" thick pillow. The pillows, covered in micro suede, are removable from the swing.
- As an extra measure of comfort, the seat area of the swing has a flat panel that fully supports the pillow. The removable panel fits in between the inner and outer layer of materials.
- The same type of spring clip connector that is used on the platform seat / board is used here for easy installation / removal.
- The Reagan's Ride swing has 6" total range of adjustment, up and down, to accommodate the optimal height of usage. The adjustment is independent front to back, allowing the swing to have some backward tilt (if desired).

## SWING ASSEMBLY / DISASSEMBLY DETAILS

The swing comes assembled out of the box; no assembly necessary. If it becomes necessary to disassemble the swings, the following steps should be followed:

- Remove the spring connectors from the chains.
- Directly at one end of the horseshoe shaped structure, pull the micro suede cover towards the swing inside, exposing the end of the tube. *See Figure 1.*
- Grasp the end of the tube with one hand, and the upper part of the swing in the other. Carefully pull the tube out of the swings pocket, feeding all the chains through their individual openings as the tube is being pulled out. *See Figure 2.* After removing the horseshoe shaped tube, set it aside.
- The flat panel under the cushion can be removed by removing the cushion first and setting it aside.
- Reach down inside the swing and grasp the panel at the back edge. Tilt the panel up until it is vertical. You should see an open pocket where the panel can be removed. The panel is a close fit in this pocket; but with a little effort, the panel can be removed up and out the swing. *See Figure 3.*
- To reassemble, just reverse the above steps.



*Figure 1*



*Figure 2*



*Figure 3*

**CLEANING**

Both layers / types of materials can be spot cleaned with water based products. The micro suede pillow cover (zippered) is removable. The pillow is moisture resistant. Do not soak the micro suede fabric, as it can cause water spots that are difficult to get rid of. After spot cleaning the micro suede, the material can be brushed to raise the nap.

There are several sites on the internet that give more detail on cleaning micro suede and woven nylon.



601 FlagHouse Drive | Hasbrouck Heights, NJ 07604 | 800.793.7900 | [flaghouse.com/haleysjoy](http://flaghouse.com/haleysjoy)



235 Yorkland Boulevard | North York, ON M2J 4Y8 | 800.265.6900 | [flaghouse.ca/haleysjoy](http://flaghouse.ca/haleysjoy)